BES 120

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Table of contents

Preface	8
Declaration of conformity	8
Service life	8
Disposal	8
Identification plate	8
Reuse	8
Warranty	8
Parts of the product	9
Intended purpose	9
Indication	9
Scope of delivery	9
Safety instructions	9
Assembly	10
Securing in the bathtub	10
Swiveling the seat	10
Use	10
Cleaning Maintenance	10
Technical data	10

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

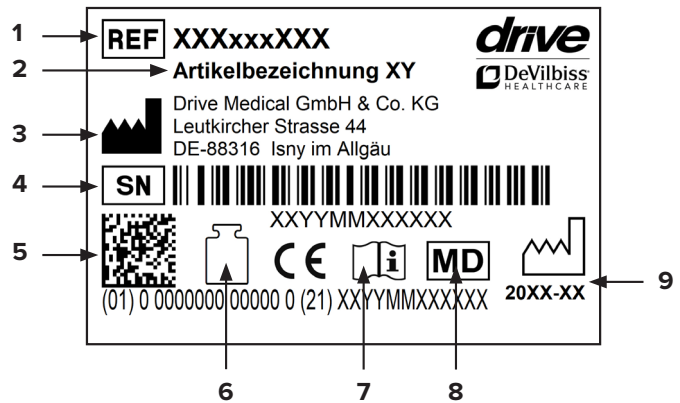1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others):
Protective cap/tub support

Parts of the product

BES 120

1. **Seating and swiveling unit**
2. **Armrests**
3. **Protective cap/tub support**
4. **Fixing screw**

Intended purpose

Bath seats hook over the bath or are placed on the edge of the bath. This reduces the distance you need to raise/lower yourself using your own strength. They are used when there are significant functional limitations of the upper and/or lower extremities. With the aid, bathing is a largely independent task again. However, using this aid presupposes that there is sufficient residual functionality in terms of torso position and use of the extremities to be able to change position.

Indication

INDICATION:

Impossibility to be securely seated or sit down and/or impossibility to maintain seated position and, if applicable, limitation in independently getting into/out of the bathtub and/or due to significantly reduced function or lack of function of the lower extremity with limited or absent mobility and muscle strength and/or significantly limited control of the torso with simultaneously preserved adequate residual functions of the upper extremities.

Scope of delivery

1 x	seating and swiveling unit
2 x	support arms, adjustable
4 x	retaining clips for the support
1 x	instructions for use

Safety instructions



- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Proceed slowly and carefully when sitting down. Do not drop!
- Caution – risk of slipping/falling!
- Do not stand on the bath seat.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in

these instructions for use as well as on the identification plate.


- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Assembly

Push the left and right support arms into the mount; in doing so, select the first hole for an upper tub edge width up to 63 cm or the second hole for a tub edge width up to 58 cm (**photo 1**). Now insert the clips into the holes, as shown in **photo 2**. Ensure that all 4 clips are inserted (**photo 3**).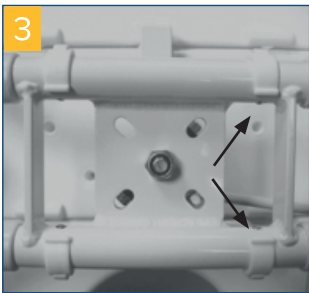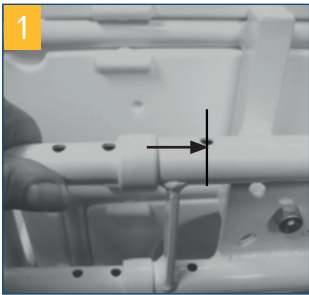

Securing in the bathtub

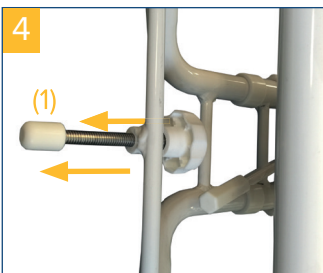Rotate both set screws (1) until they touch the edge of the tub and prevent the seat from slipping (**photo 4**).

The edge of the tub must be clean and dry. Always adjust the width of the supports to the maximum width in order to guarantee secure support.

Swiveling the seat

The seat can be swiveled after pulling the locking lever (2) (**photo 5**).

This function should be checked prior to initial use.

Use

- Hold onto the armrests securely with both hands and sit on the seat.
- Now pull the locking lever (2) upwards, raise your legs and swivel them into the tub (**photo 5**).
- The released locking lever automatically clicks into all 90° positions. **Do not press the locking lever downwards!**
- To get out of the tub, follow the steps in reverse order.

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

Technical data

Article number	550600200
HMV (medical aids register) no.	04.40.02.3020
Max. load	120 kg
Seat width x seat depth	41 x 41 cm
Overall width	78 cm
Height of backrest	40 cm
Seat adjustment	4 x 90°
Color	white
Total weight	4,5 kg
Frame material	Aluminium, powder-coated
Seat material	Plastic (PE-HD)

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Drive Medical GmbH & Co. KG