AiO

Inhalt

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Contents

Preface	9
Declaration of conformity	9
Service life	9
Disposal	9
Type plate	9
Re-use	9
Warranty	9
Applicable standards	9
Parts of the product	10
Purpose	10
Indications	10
Scope of supply	10
Safety notes	10
Installation	11
Adjusting the height of the feet	11
Preparation for re-use	12
Cleaning Maintenance	12
Technical specifications	12

Preface

Thank you for choosing a Drive DeVilbiss product.

The design, functionality and quality of this product will not disappoint you.

Please read these operating instructions carefully before using the product for the first time. They contain important safety notes and valuable tips for using and caring for your product properly. If you have any questions or need additional information, please contact the Drive DeVilbiss dealer who delivered the product to you.

These operating instructions contain all the necessary information for adjusting and using the product. The latest operating instructions can also be downloaded as a pdf online or can be requested from Customer Service.

Large format documents can be displayed for visually-impaired individuals. Blind patients must be trained by their assistant. Repairs as well as some adjustments require special technical training and must therefore be carried out by a specialist medical company.

Declaration of conformity

DRIVE MEDICAL GMBH & Co. KG declares conformity for the product described in accordance with the Medical Devices Regulation (EU) 2017/745.

Service life

Our company expects this product to have a service life of five years, as long as the product is used for its intended purpose and that all maintenance and service instructions are heeded.


This service life may be significantly longer, if the product is handled carefully and is serviced, looked after and used properly and there are no technical limits in the development of science and technology.

However the service life can also be considerably reduced through extreme usage and incorrect use. Definition of the service life by our company does not constitute any additional warranty.

Disposal

If you no longer need the product, please contact your dealer or take the product to your local disposal centre.

Type plate

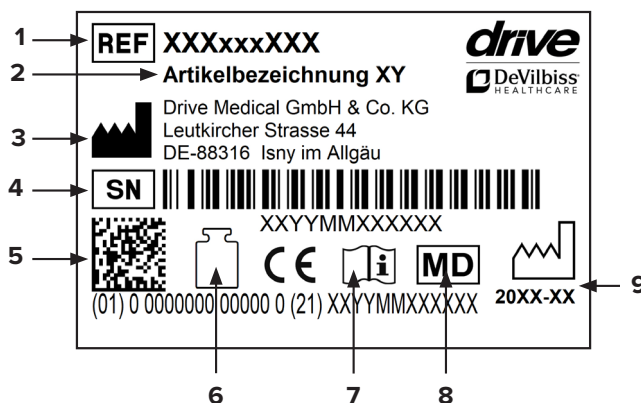1 Item number | 2 Item description | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. Loading | 7 observe instructions for use | 8 Medical product | 9 production date (year-month)

Re-use

This product is designed to be re-used. Make sure that all the technical documentation needed for safe handling are passed on to a new user.

The product must be checked and cleaned by the retail trade who will restore it to perfect working order ready for re-use.

When preparing the product, the manufacturer's instructions must be observed and these must be made available on request.

Please ask your dealer or see www.drivedevilbiss.de for available accessories

Warranty

The warranty period for the product we have delivered is 24 months from the date of purchase. Provided that it can be proven, if there is a material or manufacturing fault, faulty parts will be replaced free of charge.

Parts subject to wear are generally not covered during the normal period of warranty, unless items require repair or replacement and this is necessary as a clear direct result of a manufacturing or material fault.

We reserve the right to make changes due to technical improvements and design changes.

EXCLUSIONS FROM WARRANTY

Parts subject to wear (amongst others): Rubber feet

Applicable standards

ISO 17966:2016 Assistive products for personal hygiene that support users.

Parts of the product

AiO

1. **Seat plate**
2. **Adjustable foot**
3. **Rubber foot**
4. **Armrests**
5. **Backrest**

Purpose

Shower aids are shower seats, stools, chairs, shower supports, mobile shower supports which are intended exclusively for domestic use. Shower aids are used for those whose physical functional limitations no longer allow daily personal care under the shower without the use of additional aids. Shower aids are prescribed, when the user's stability is impaired. If there is also a reduction in seating stability (e.g. paralysis in the trunk) or a weakness when sitting down/standing up, then backrests and side supports must also be used if necessary.

Indications

For individuals who cannot stand safely in the shower due to damage to the function of the lower extremities with limited or failing mobility and muscle strength and/or damage to the vestibular functions or tendency to fall for other reasons while at the same time having a sufficiently stable torso position to be able to sit and sufficient residual functionality in the upper extremities.

Contraindication

The use of the product is not indicated for Dizziness or inability to sit.

Scope of supply


1 x	Seat plate
4 x	Adjustable foot
2 x	Armrests (included with Item no. 550900200, 550900400)
1 x	Backrest (included with Item no. 550900300, 550900400)
1 x	Instructions for use

Safety notes

- Read the instructions for use carefully.
- Before use check all components for damage and make sure they are secure. Do not use the product if it is defective!
- When sitting down, standing up and when clicking the seat plate into position into place, clothing or body parts could get caught.
- Caution - Risk of slipping/falling!
- The tipping stability of the product is significantly lower than with standard chairs, please therefore avoid leaning to the side – risk of tipping!

- Always sit in the centre of the seat surface or right at the back next to the backrest and not on the edge – risk of tipping or slipping!
- Caution – Risk of the fingers getting pinched during assembly
- The product must not be exposed to temperatures over 60° C for a sustained period (e.g. use in a sauna, etc.).
- Observe the maximum permitted weight for the user. This information can be found within the technical data in these operating instructions as well as on the type plate.
- Any serious incident that occurs with the product must be reported to the manufacturer and the relevant authorities of the member state in which the user and/or patient is domiciled.

Installation

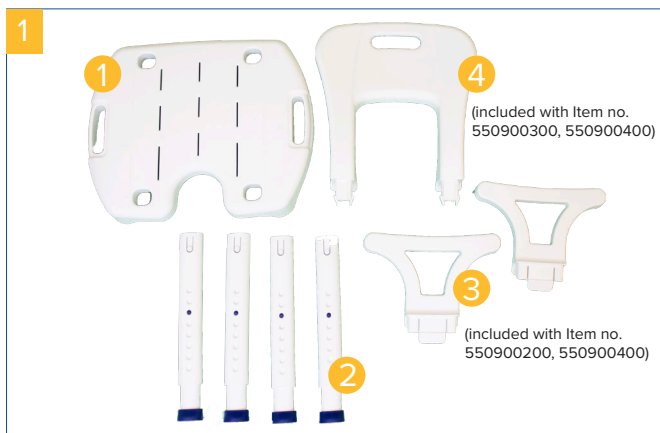1. Fixing the chair legs

Place the seat plate (1) on a clean flat surface with the underside facing upwards. If necessary, use a soft pad to prevent scratches. Press the white knob at the top end of the chair leg (2) and push it into the matching recess in the seat plate (Fig. 2).

The chair leg is properly fixed when you hear a “click” and the button is fully out (Fig. 3).

2. Fixing the armrests

(included with Item no. 550900200, 550900400)

Push the armrest(s) (3) into the recess in the seat plate until the interlocks click into place.

3. Fixing the backrest

(included with Item no. 550900300, 550900400)


Press the backrest (4) into the front or rear recesses of the seat plate until the interlocks click into place.

Adjusting the height of the feet

Before using the product adjust it to the height required.

To adjust the height of the legs: Press the button (1) and at the same time pull the leg out or push it in until the knob clicks into place in the position required (2). Make sure that the push button is properly locked into place.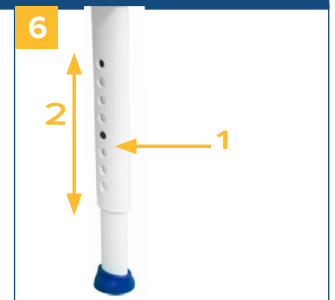

Make sure that all four legs are adjusted to the same height.

Caution - risk of tipping over!

Preparation for re-use

- Check that the product is intact and clean it
- Observe the information for re-use (page 3)

Cleaning | Maintenance

- Maintenance-free
- Clean regularly with a damp cloth by hand.
- Do not use any aggressive detergents!
- Do not use any abrasive detergents!
- Do not use any high-dose chlorine or vinegar-based cleaners!
- For disinfection you can use a spray disinfectant, e.g. Descosept AF or Bacillol AF.
- If there are defects, call the dealer from whom you bought this product.

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Drive Medical GmbH & Co. KG