DKS 130

Inhalt

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Table of contents

Preface	7
Declaration of conformity	7
Service life	7
Disposal	7
Identification plate	7
Reuse	7
Warranty	7
Parts of the product	8
Indication	8
Scope of delivery	8
Safety instructions	8
Assembly	9
Height adjustment of the feet	9
Compensating for uneven surfaces	9
Folding up and down	9
Cleaning Maintenance	9
Technical data	9

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

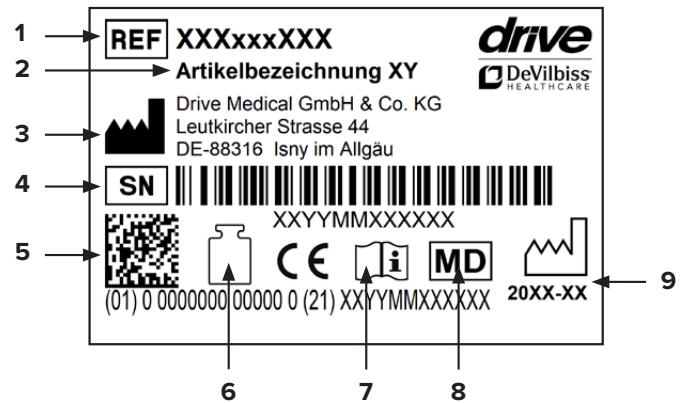1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others):
rubber feet

Parts of the product

DKS 130

- | | |
|---------------------------|--|
| 1. Seat section | 4. Fixation clamp for height adjustment |
| 2. Mounting plates | 5. Finely adjustable foot |
| 3. Frame | 6. Balancing screw |

Indication

INDICATION:

Impossibility to securely stand in the shower due to impaired function of the lower extremity with limited or absent mobility and muscle strength and/or impaired vestibular function or tendency to fall due to other etiology with simultaneously adequate stable torso position for sitting and sufficient residual functions of the upper extremities.

Scope of delivery

1 x	folding shower seat
8 x	end caps
1 x	instructions for use
1 x	drilling template

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Clothing or body parts can be pinched when sitting down or getting up and when locking the seat section into place.
- Caution – risk of slipping/falling!
- The product should not be exposed to temperatures above 60 °C for long periods of time (for example, use in a sauna, etc.).
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.



Assembly

The seat height must be determined and adjusted by a professional.

- Both of the user's feet must be on the floor while seated.
- Take a regular chair on which the user can sit comfortably.
- Have the user sit on the chair.
- Measure the height of the seat of the chair.
- From the adjustable measurements of the folding shower seat (465, 490, 515 or 540 mm), select the measurement which is closest to the measurement determined and transfer this measurement from the floor of the shower basin to the wall.
- Cut a window in the drilling template around the center line
- Lay the drilling template in position and transfer the 8 drill holes to the wall. If possible, use the gaps between tiles for drilling to avoid damaging the tiles.
- Drill the holes and insert the anchors.


Height adjustment of the feet

To adjust the height of the footrests, first loosen the fixation clamp (**pic. 1**).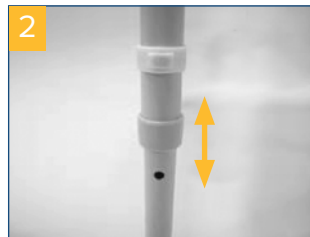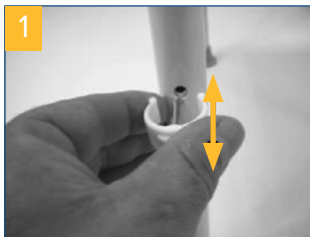


Then the footrests can be adjusted to one of 4 heights (**pic. 2**), for seat heights of 465, 490, 515 and 540 mm.

Compensating for uneven surfaces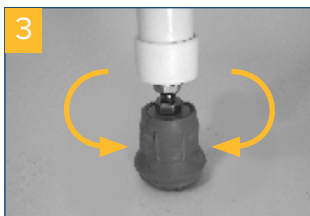

To compensate for any unevenness in the shower, one foot is equipped with fine adjustment (**pic. 3**).

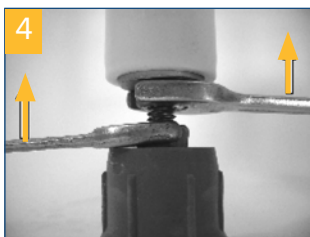Adjust the foot on the floor by screwing it in or out and then lock the set screw with the two locking nuts as shown (**pic. 4**).

Folding up and down

To fold the folding shower seat up or down when not in use, always grip the seat (**pic. 5**) only, never as shown in **pic. 6** - Risk of pinching!

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- In the event of defects, contact your specialist dealer from whom you purchased this product.

Technical data

Product	DKS 130
Article number	550700100
HMV (medical aids register) no.	04.40.03.0029
Overall width	48 cm
Overall height	61 cm
Overall depth	50 cm
Seat width x Seat depth	41 cm x 41 cm
Seat height	46,5 - 54 cm
wall clearance / seat depth to the wall	32 cm / 49 cm
Total weight	4,1 kg
Load max.	130 kg
Material frame	Welded steel tubing, powder-coated
Material seat section	PP (polypropylene) / PU (polyurethane)

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Drive Medical GmbH & Co. KG