

PRODUCT INSTRUCTION SHEET

REFERENCE 172 (Uppingham combined shower and commode chair)
 172-LONGARMS (Uppingham combined shower & commode chair (LONG ARMED VERSION))
 172WH (Footrest)

DESCRIPTION

Manufactured from peel-resistant, plastic-coated steel tubing, this very robust and stylish product is supplied with a comfortable PU (polyurethane) cellular moulded gap-fronted seat, backrest, over seat and padded hinged armrests.

The unit features a pushing handle, 4 braked nylon castors (125mm diameter) and a commode bowl. Also included is a rota moulded plastic detachable footrest with 2 height positions. The footrest clips to the frame and retracts when not in use. All accessory components are made from stainless steel for product longevity with re-usability and re-issue in mind.

The Uppingham is an aid for able bodied people to transport users/occupants quickly and safely between locations and to allow for easy manoeuvrability of the chair when unoccupied.

TECHNICAL SPECIFICATIONS

- Width between arms: 21" (54cm)
- Seat height: 20" (50cm)
- Seat width: 17" (42cm)
- Seat depth: 17" (42cm)
- Aperture size: 11.5" (33cm) x 8.5" (22cm)
- Depth of pot: 7" (18cm)
- Seat height: 20" (51cm)
- Seat height w/o cover: 18" (45.7cm)
- Overall width: 25" (64cm)
- Overall depth: 21" (52cm)
- Height to push handle: 40" (102cm)
- Weight limit: 160kg (25 stone)
- Arm length: 13" (33cm) or 16" (41cm)





PRODUCT INSTRUCTION SHEET

SAFETY USAGE

- Ensure castors are fully secure before and after use. Castor security must be checked periodically.
- Appliances to be used on a level surface only.
- Do not exceed the weight limit of 165kg (26 stone) or swing on the frames in any direction. The four castors must always be in contact with the floor.
- When issuing a shower chair, a risk assessment should be carried out to determine the suitability of the equipment for the user, with emphasis on the risk of entrapment.
- The footrest has 2 height positions; the higher position must only be used for children or smaller adults to prevent tipping of the unit when in use.
- Ensure the spindle bolt which holds the armrests is tightened at all times.
- If using a moulded footrest, make sure the sides are properly located on the bottom struts of the frame, and move freely from front to rear and vice versa.
- When in use, all four brake castors should be secured at all times. The front two castors should have the brake pedal facing forward, and parallel with the side of the chair. This will prevent the chair from tipping if the client leans forward.

CLEANING AND CARE

- This product is intended for use in a wet room or shower.
- It is not intended for use outside and should not be immersed in water for long periods of time.
- The chair is designed to withstand commercial washing processes up to 80°C. Alternatively, the chair can be cleaned with a soft cloth, warm water and mild detergent. Chlorine solution up to 1000 ppm (parts per million) may also be used.
- Ensure the product is dried thoroughly after use, or after being cleaned.

WARRANTY

Your Drive DeVilbiss Healthcare branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment.

This warranty is an expression of our confidence in the materials and workmanship of our products, and our assurance to the customer. In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive DeVilbiss stockist. This does not affect your statutory rights.