TSE 150 / TSE 150 Plus

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Table of contents

Preface	9
Declaration of conformity	9
Service life	9
Disposal	9
Identification plate	9
Reuse	9
Warranty	9
Parts of the product	10
Intended purpose	10
Indication	10
Scope of delivery	11
Safety instructions	11
Use	11
Assembly	11
Height settings	12
Cleaning Maintenance	13
Technical Data	13

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate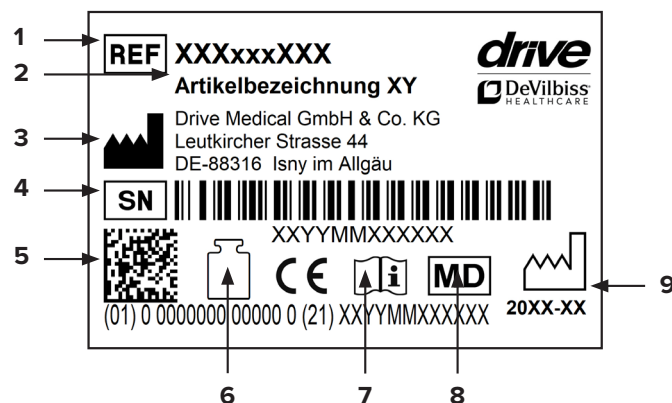

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year/month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available at www.drivedevilbiss.de or from your specialist dealer.





Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

Parts of the product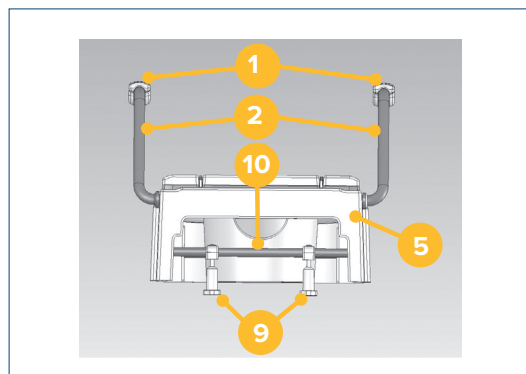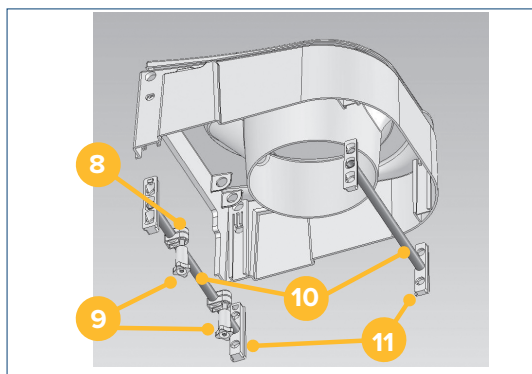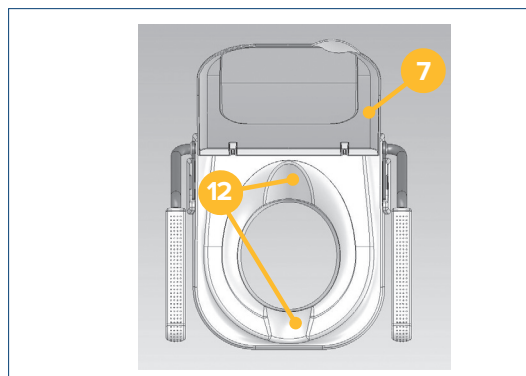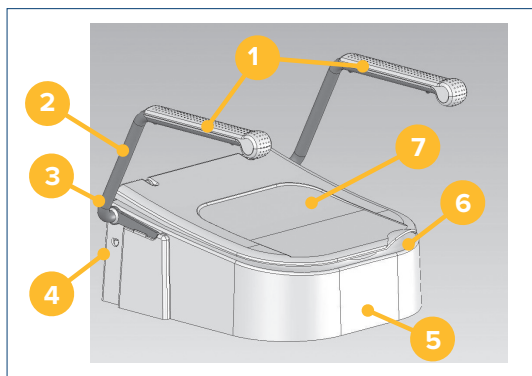

TSE 150

- | | | |
|---|--------------------------------------|-------------------------------|
| 1. Armrest cover | 5. Body | 9. Knurled nuts |
| 2. Armrest | 6. Recessed grip (in the lid) | 10. Crossbars |
| 3. Armrest holder | 7. Lid | 11. Adjustable holders |
| 4. Locking mechanism for adjustable holder | 8. Clamp | 12. Hygiene openings |

Intended purpose

Toilet seats are placed over the existing toilet bowl or screwed onto the toilet and are used to raise the height of the toilet. Toilet seat raisers can be used with and without armrests. Toilet aids enable or increase independence when using the toilet and the related personal hygiene, particularly because they compensate for the impaired or missing functions of the motor system or muscular support structure.

Indication

INDICATION:

Use of the toilet hindered due to unadapted seat height through limited ability to sit down and stand up along with limited control of the trunk with functional disorder of the lower extremity with significantly restricted or fully absent movement of the joints and/or limited muscular strength of the lower extremity and/or damage to the lumbar spine, hip and/or knee and/or damage to the nervous system/central nervous system and/or damage to muscle function with limited or absent stability of the trunk.

CONTRAINDICATIONS:

The toilet seat raiser is to be used exclusively to raise a toilet seat and is therefore designed to fit onto a toilet bowl. Any use other than the use stated is not permitted. The toilet seat raiser is only designed for adults. Do not use as a child's toilet seat. With correct use, there are no contraindications.

Scope of delivery

1 x A	Body
1 x B	Crossbar rear
4 x C	Adjustable holder
1 x D	Crossbar front
2 x E	Armrest (only for 5201000)
2 x F	Knurled nuts
2 x G	Threaded rods M8
2 x H	Star screw (only for TSE 150 Plus)
2 x I	Top-mounting pin (only for TSE 150 Plus)
1 x	Instructions for use
1 x	Vinyl gloves

Safety instructions

- Do not use if there are any defects!
- Caution – risk of slipping/falling!
- The lid must not be used as a backrest in the opened position!
- The lid must not bear a load when closed (by sitting on it, for example)!
- Check the raised toilet seat regularly for a secure fit on the toilet bowl. Retighten the knurled nuts if necessary.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Use

The raised toilet seat TSE 150 is suitable for nearly all standardized or commercially available toilet bowls.


Assembly

Remove the seat and lid from the toilet bowl. Assembly is carried out without tools.

TSE 150:

Remove the hinge and lid from the toilet bowl. Assembly is carried out without tools. To do so, set the raised toilet seat on the toilet bowl, adjust both threaded rods M 8 (**G**) after removing the knurled nuts (**F**) to the appropriate width and insert through the holes of the toilet bowl. Depending on the length of the toilet bowl, it should be ensured that the clamps on the rear crossbar (**B**) are adjusted in the corresponding direction such that support of both crossbars is ensured. Screw both knurled nuts (**F**)

loosely back onto the threaded rods (**G**), align the raised toilet seat, and then tighten both knurled nuts (**F**). Check all screws for tightness and tighten if necessary. Only the supplied fastening materials may be used for assembly.

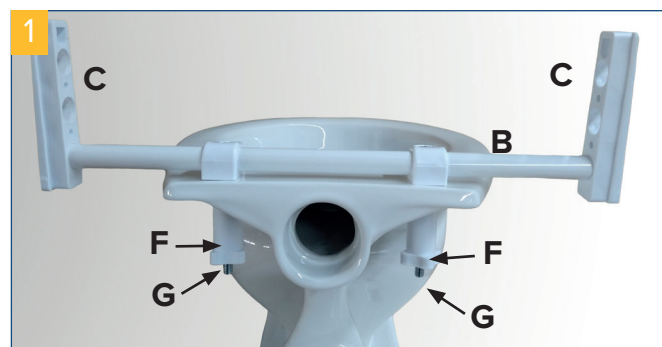
TSE 150 PLUS:

Push the top-mounting pin together (**I**) as shown in the photo and insert down into the mounting holes of the ceramic bowl (**pic. 2**).

Place the rear of the support (**B**) on the ceramic bowl and screw in the star screws (**H**) from above but do not tighten them yet. Align the rear of the support (**B**) in the center on the ceramic bowl and tighten the star screws (**H**)(**pic. 3+4**).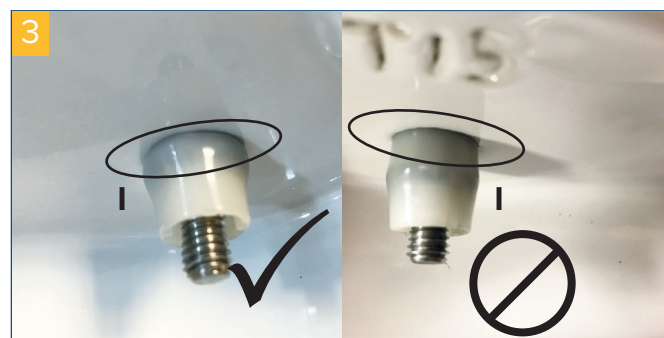

Insert the adjustable holders (**C**) at the desired height (6,5 / 10 / 13,5 cm) on the rear of the support (**B**) on the right and left (**photo 4**).

Insert the adjustable holders (C) at the desired height (6,5 / 10 / 13,5 cm) on the rear of the support (D) on the right and left.

Take care that the front supports (D) on the right and left are each inserted at the same height.

The front supports (D) can be set lower than the rear support (B).


Insert body (A) of the raised toilet seat into the adjustable holders (C). In doing so, the adjustable holders (C) must engage with the locking tabs (L) in the body (A).

The adjustable holders (C) can only be inserted in one direction into the raised toilet seat.

LID ASSEMBLY:

The lid is preassembled at the time of delivery, but can be replaced if necessary.

To do this, fold the lid all the way back. The lid can then be taken rearwards out of the holders. Insert the new lid accordingly. The lid can then be taken out of the holders. Clip the new lid on.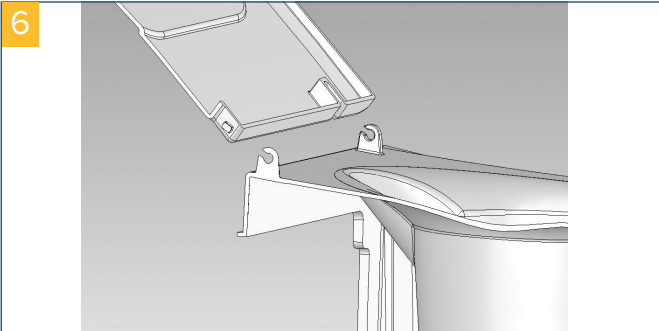



ARMREST ASSEMBLY:

Push the armrests sideways into the armrest holders provided for this purpose.


Now push the armrests (E) in until they catch, and then fold them down if needed so they rest on the support

Height settings

The raised toilet seat can be adjusted using the adjustable holders for an elevation of 6.5 cm, 10 cm or 13.5 cm (pic. 7 + 8).

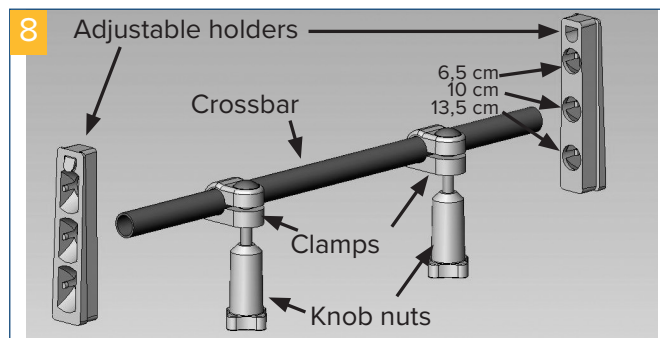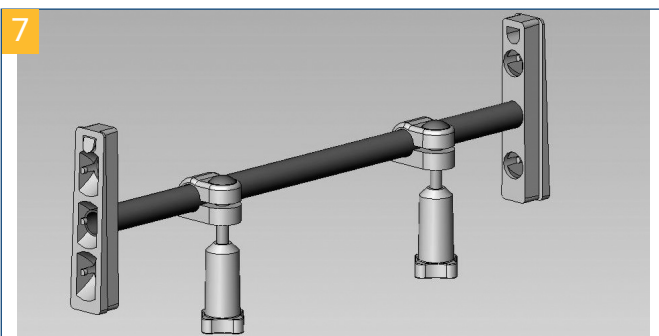
It is set at 10 cm at the time of delivery. (pic.9).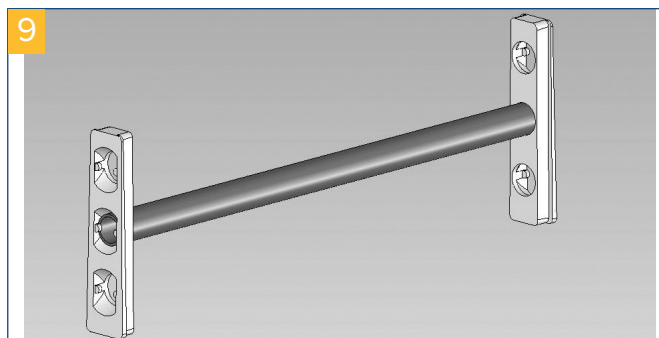

If you wish to change this, follow these steps:

- Lightly push the locking tabs on the right and left under the armrest holders and pull out the rear crossbar with the adjustable holders. Pull out the front crossbar with the adjustable holders likewise.
- Adjust the desired front and rear height (pic. 10).

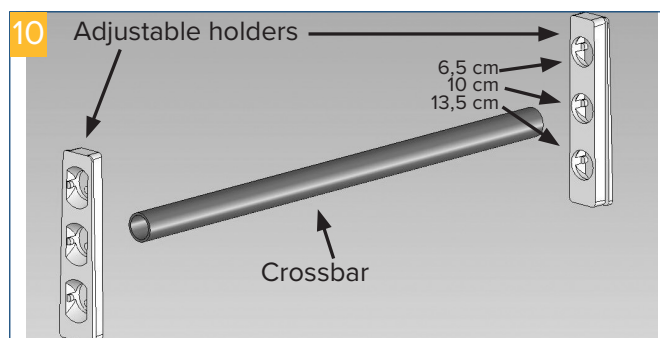- To do this, insert the crossbars in the corresponding opening in the adjustable holder.
- Take care that the crossbars on the right and left are each inserted at the same height.
- The front crossbar can be set lower than the rear crossbar.
- Insert the front and rear crossbars with holders back into the raised toilet seat. In doing so, the holders of the rear crossbar must engage with the locking tabs

The adjustable holders can only be inserted in one direction into the raised toilet seat.

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

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Drive Medical GmbH & Co. KG