CROW

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Table of contents

Preface	16
Declaration of conformity	16
Service life	16
Disposal	16
Type plate	16
Re-use	16
Warranty	16
Parts of the product	17
Scope of supply	18
Safety notes	18
Introduction	19
Purpose	19
Electromagnetic compatibility (EMC)	19
Control panel	19
Use	19
Battery	21
Tyres	22
Driving on the road	22
Dismantling	22
Storage	23
Cleaning Disinfection	23
Troubleshooting	24
Inspection and servicing	25
Guarantee	25
Technical specifications	26
Serial number	27
Maintenance report	27

Preface

Thank you for choosing a Drive DeVilbiss product.

The design, functionality and quality of this product will not disappoint you.

Please read these operating instructions carefully before using the product for the first time. They contain important safety notes and valuable tips for using and caring for your product properly. If you have any questions or need additional information, please contact the Drive DeVilbiss dealer who delivered the product to you.

These operating instructions contain all the necessary information for adjusting and using the product. The latest operating instructions can also be downloaded as a pdf online or can be requested from Customer Service.

Large format documents can be displayed for visually-impaired individuals. Repairs as well as some settings require special technical training and must therefore be carried out by a specialist medical company.

Declaration of conformity

The manufacturer declares conformity for the product described in accordance with the conditions of the 93/42 EEC Directive from the council from 14.06.1993 for medical products and the 2007/47/EC Directive.

Service life

Our company expects this product to have a service life of five years, as long as the product is used for its intended purpose and that all maintenance and service instructions are heeded.


This service life may be significantly longer, if the product is handled carefully and is serviced, looked after and used properly and there are no technical limits in the development of science and technology.

However the service life can also be considerably reduced through extreme usage and incorrect use. Definition of the service life by our company does not constitute any additional warranty.

Disposal

If you no longer need the product, please contact your dealer or take the product to your local disposal centre.

Type plate

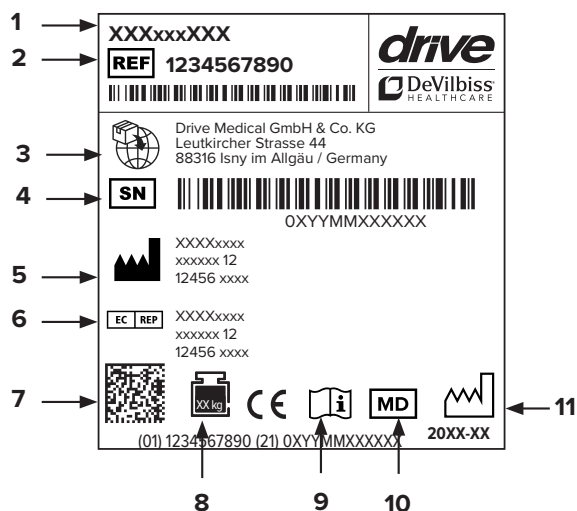1 Article number | 2 Article description | 3 Importer | 4 Serial number | 5 Manufacturer | 6 EC-REP | 7 UDI-Code | 8 Max. Load | 9 Observe instructions for use | 10 Medical device | 11 Production date (year/month)

Re-use

This product is designed to be re-used. Make sure that all the technical documentation needed for safe handling are passed on to a new user.

The product must be checked and cleaned by the retail trade who will restore it to perfect working order ready for re-use.

When preparing the product, the manufacturer's instructions must be observed and these must be made available on request.

Please ask your dealer or see www.drivedevilbiss.de for available accessories

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase and 6 months for the battery from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others):
Tires, handles, cushions, batteries, carbon brushes

Parts of the product

**Crow**

1. Seat
2. Armrest
3. Adjusting the armrest extension
4. Seat adjustment lever
5. Side reflectors
6. Battery
7. Cover for the safety switch
8. Charging socket
9. Bumper
10. Headlight
11. Steering column adjustment
12. Basket
13. Cockpit
14. Rear light
15. Rear reflectors
16. Freewheel lever
17. Anti-tip aid

Scope of supply

1 x	Electric scooter
1 x	Charger including instructions for use
1 x	Instructions for use

Safety notes

Your scooter is a powerful machine. Please read the following notes through carefully, to safeguard yourself at all times.

General

- Read the instructions for use carefully.
- Before use check all components for damage and make sure they are secure. Do not use the product if it is defective!
- Never use the scooter if you are under the influence of drugs, alcohol or certain medication which impair your safety.
- Never use electronic radio devices such as walkie-talkies or mobile telephones.
- When you are driving backwards make sure that there are no obstacles behind you.
- Do not change course suddenly at full speed especially on uneven or hilly surfaces. Failure to comply with this can lead to the scooter tipping over.
- Use your scooter in public traffic, where possible using the cycle path.
- Do not drive over kerbs where the height of the kerb is greater than the ground clearance of your scooter. Approach the kerb slowly and at a slight angle, hold the handlebar firmly with both hands while doing this.
- When driving the scooter always keep both hands on the steering system and your legs in the scooter.
- Do not drive your scooter in snow, in order to avoid accidents on slippery roads.
- Only change the direction of travel (forwards, backwards) when at a standstill.
- Do not carry any passengers and do not exceed the maximum weight. (Please also see the Technical Specifications for your model).
- Do not switch your scooter on if the forwards/backwards lever is pressed.
- Never get on or off your scooter if the scooter is not switched off.
- Always switch the scooter "off" before you swivel the seat.
- In narrow spaces always drive at the lowest speed setting.
- Make sure there is sufficient space to turn

in narrow streets and corridors. Due to the design, problems may occur when turning the scooter.

- Only drive your scooter when the seat is clicked into place in the drive position.
- Never drive your scooter over deep, soft ground (such as, for example, sand, long grass, loose gravel).
- Do not drive with your scooter on uphill or downhill slopes, which exceed 16%.
- Always proceed carefully when you are driving your scooter, especially when you are near a slope.
- Always proceed carefully when you are driving over uneven surfaces.
- NEVER drive your scooter through deep water and never clean your scooter with a pressure washer.
- Never expose the scooter to direct sunlight or other heat sources over a longer period of time. There is a possible risk of burns from the seat surface and metal parts.
- Never expose yourself and the scooter to extreme cold over long periods of time. Damage to the user or scooter cannot be ruled out.
- Do not allow children to play unsupervised near the device when the batteries are being charged.
- Never exceed the maximum load limit for your vehicle.
- Observe the user's maximum permitted weight. This information can be found within the technical data in these operating instructions as well as on the type plate.

Batteries

- Do not drop, bang, crush or otherwise misuse the batteries as this could lead to the corrosive battery contents being exposed.
- In the event of a battery leaking, do not allow the liquid to come into contact with the skin or eyes. If there is contact, rinse the affected area with plenty of water and seek medical advice.
- Do not expose the batteries to fire or extreme heat. Do not incinerate. Do not expose the battery to intense heat as this could cause an explosion. Avoid storing in direct sunlight.
- Do not expose the batteries to water, rain or any type of moisture.
- Do not crush, dismantle, pierce or shortcircuit the connecting terminals.
- Do not try to open, dismantle or repair the batteries; they do not contain any repairable parts.
- Do not simply store batteries in a container, where

- they could shortcircuit against one another or could be shortcircuited by other metal objects.
- Keep batteries away from children.
- Keep batteries clean and dry.
- Only use the batteries for the application for which they are intended.
- Check the connecting cable, connection heads and power supply regularly to make sure the connections are secure, check for damage or signs of wear. Do not use if damaged.
- Charge the batteries fully before using for the first time.
- Max. Time period between charges = 6 months.
- Recommendation: Store the batteries below 25 ° C, with low humidity, in a dustfree and non-corrosive gas atmosphere. Where possible, store the batteries fully charged.
- This device contains electrical and/or electronic components. Observe local government and recycling regulations to dispose of the device components.
- Dispose of the batteries properly.

Introduction

Please read this user handbook carefully before you use the scooter. Incorrect use of the scooter can lead to damage, injuries or traffic accidents.

This handbook explains how you can use your scooter in the best way. It includes instructions for the operation, assembly and maintenance of the scooter.

A repair and maintenance table and warranty information are also included in this handbook. Please keep the handbook in a safe place or store on the scooter.

Should other individuals use the scooter, then please make sure that the handbook is passed to these individuals for review.

As the design is subject to change, some of the illustrations and photographs in the handbook may possibly not match the scooter that you have purchased. We reserve the right to make design changes.

The scooter has been designed and constructed, to offer a convenient and safe, yet affordable solution for specific mobility requirements.

Purpose

The scooter has been designed for outdoor use for individuals with limited walking ability. It is classed as a category C product in accordance with DIN EN 12184.

Electromagnetic compatibility (EMC)

This product has been tested for electromagnetic compatibility (EMC) in accordance with IEC 60601-1-2 and ISO 7176-21

Control panel

1. Horn
2. Control and warning light
3. Battery display
4. Drive lever
5. Speed controller
6. Ignition switch

Use

Ignition key

To start the scooter, turn the key to the right; the control panel will now light up. To switch it off, turn the key to the left; the control panel will switch off and you can pull the key out.

Warning: Do not switch the ignition off while you are driving, as this will cause an immediate stop and possible damage or injury.

Drive lever

The scooter is operated by the driver moving the drive lever to the side on the control panel.

To drive the scooter forwards, move the right drive lever in the direction of the user. To drive the scooter backwards, move the left drive lever in the direction of the user. The horn emits a beep when the vehicle is going backwards. The further the drive lever is moved, the faster the scooter will drive.

To stop the scooter, release the drive lever; the electro-magnet brake will activate automatically.

Warning: Never operate the left and right drive lever at the same time. This would mean that it is no longer possible to control the scooter.

Brakes

By releasing the drive lever, the vehicle will brake automatically. When the vehicle comes to a standstill the magnetic brake will be applied automatically. The scooter is now secured against rolling away.

Warning: Never switch the vehicle to freewheel on slopes. The motor brake will not take effect in these circumstances.

Adjusting the speed

The speed of the scooter can be increased by turning the speed controller to the right (hare symbol), or reduced by turning the speed controller to the left (tortoise symbol).

Warning: Do not adjust the speed setting while driving, as this could lead to a loss of control. Do not select the highest speed when you are driving indoors.

When the scooter is being used indoors or in pedestrian areas (e.g. on pavements or in shopping areas) you should select the lower speed setting. When the scooter is being used on the road, you can select the high speed setting.

Warning: Always use the lowest tempo setting when you are driving indoors or on the pavement.

Horn

When the "horn" button is pressed, a beep will sound, as soon as the button is released, the beep will stop.

Seat


The seat can be rotated by up to 360° and clicks into place in 45° steps. To rotate, pull the lever (A) under the right side of the seat upwards and then rotate the seat. Release the lever and then continue to rotate the seat until it clicks into place.

Removing the seat: Pull the lever A upwards and lift the seat up and off at the same time.

Adjust the seat height:

- Take the seat off your scooter.
- Remove the locking pin B from the lower seat support.
- Adjust the locking pin B to the required seat height.
- Move the upper seat support such that the hole for inserting the locking pin is clear.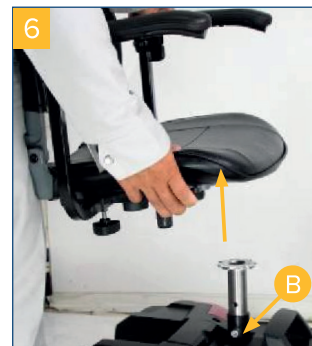

Fitting the seat: Before the seat is put back in place, pull the lever A back up and fit the seat properly. Then release the lever A again. When fitted correctly, the seat must click back into place at 45° steps.

Important: You must hear and feel the seat click into place

Caution: Move the seat to the forward-facing position before you drive.

Adjusting the armrest

First undo the star screws on the underside of the seat frame (A), then adjust the gap between the armrests by moving them. As soon as you have the right gap, then retighten the screws.

Adjusting the tiller

The tiller can be adjusted to different positions:

- Undo the screw to adjust the tiller to the required position. Re-tighten the screw.

Freewheel lever

The freewheel lever allows you to disengage the automatic brake so that the scooter can be pushed and/or manoeuvred by hand. In order to be able to push the scooter it must be switched off. In freewheel mode, the brakes do not have any effect, therefore it should not be used if:

- There is someone sitting on the scooter
- The scooter is left unattended
- The scooter is on a slope


Drive mode/freewheel mode

Drive mode:

Move the lever to **N**: The scooter will be driven by the motor.

Freewheel mode

Move the lever to **D**: The scooter is in freewheel mode and can now be pushed/moved by hand.

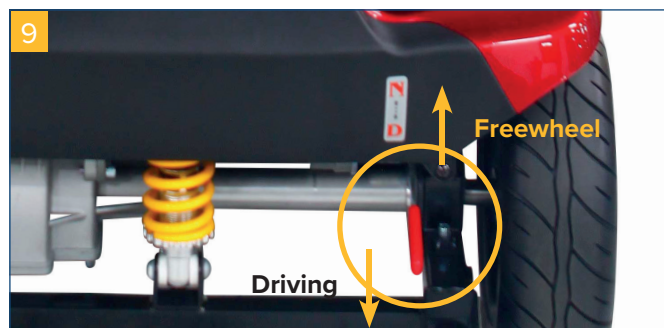Battery

Battery display

This display shows the battery status.

Green indicates remaining power of (40~100%)

Yellow indicates discharge of (10 ~30%)

Red indicates that the battery must be recharged as soon as possible.

The remaining power shown on the battery display depends on the drive time and your driving style. With repeated starts and stops and driving up slopes, the remaining energy will be used more quickly.

Charging the battery

If the scooter is not going to be used for a longer period of time, make sure that the batteries are charged at least once a month, in order to avoid deterioration of the condition of the battery.

Replacing the battery

It is normal that the battery capacity reduces over time, even if the battery has always been recharged as described above. When the battery power only reaches

60% of its previous capacity, we recommend that the batteries are replaced. Continuing to use the old batteries leads to a rapid reduction of the range of the scooter and can lead to severe wear on other parts of the scooter. It is recommended that batteries are replaced in pairs and that you ask your dealer to provide them.

Battery care

1. You should recharge the batteries every time you have used the scooter, in order to guarantee the maximum battery range. The batteries should be recharged at least once a week, even if the scooter is not being used.
2. After recharging or fitting a new battery, move the scooter a short distance, to make sure that the battery capacity is sufficient.
3. In cold environments, the battery can react more slowly and the range will be shortened accordingly.
4. When driving up or down slopes the battery display will fluctuate. This is a normal procedure.
5. The battery range will be reduced if you drive on slopes or uneven ground because the scooter will need more power.
6. The batteries should not be charged for longer than 24 hours.
7. The batteries are sealed and must therefore not be filled with water.
8. You should check the batteries regularly to make sure they are clean.

Cleaning the battery

If the batteries have been contaminated by water, battery acid, dust or other substances, they will discharge rapidly. The batteries delivered with the scooter are sealed and as such are maintenance-free and spillproof. Please follow these steps to clean the battery.

1. Set the operating switch of the scooter to "OFF".
2. Follow the instructions in replacing the battery
3. Use a clean cloth to clean the dirty surface.
4. Take the battery out.
5. Clean the battery with a clean cloth. If the terminals are covered with a white powder, please wipe this away using warm water.

Overload switch

Due to excessive loading e.g. through steep slopes, the overload switch may trip, to protect the motor and electronics from damage.

Tyres

The tyre condition is affected by how you drive the scooter and what surface it is being driven on. Please check the profile depth of the tyres regularly. Replace the tyres if the profile depth is below 1 mm.

Driving on the road

1. Check that the seat is installed correctly and is in the forward-facing position.
2. Make sure that the tiller and seat are adjusted for a comfortable driving position.
3. Flip the armrests down so that you can place your arms on them.
4. Switch the ignition on.
5. Check the battery display to make sure that you have sufficient power for your journey. If you are unsure of the residual energy, then please charge your battery before setting off.
6. Set the speed selector to a setting which is safe and comfortable for you.
7. Check that the drive lever is working properly.
8. Check that the electromagnetic brake is working properly.
9. Make sure that your immediate environment is safe before you drive on the road. Set the lowest tempo setting when you are driving on the pavement or indoors.

Caution:

1. Never press the left and right side of the drive lever at the same time. Doing this means that you cannot steer your scooter and this can lead to damage or injury.
2. Do not switch the ignition off while you are driving, as this will cause an immediate stop and possible damage or injury.
3. Do not select the highest speed when you are driving indoors.
4. Do not adjust the speed while driving. A sudden change in speed can endanger you and others and damage your scooter.
5. Do not put magnetic objects near the electrical parts of the scooter, as these could affect the safe operation of the scooter.
6. Be careful when you drive when there is a high level of traffic or in crowded areas.
7. When going backwards, be aware of people and objects behind your vehicle. When driving backwards the scooter emits an acoustic warning signal.

Stopping/parking

1. Release the drive lever completely. The vehicle will brake naturally and stop.
2. Set the switch to (OFF). Then pull the key out.

Caution:

1. When changing the forward/backwards speed, the braking distance will also change. You should therefore start braking as early as possible.



2. When you park your scooter, make sure that it is on a level surface and switch the power supply to "OFF", before you get off.

Dismantling

Follow the individual steps to dismantle the scooter:

1. Pull the stop lever under the seat upwards and lift the seat up at the same time. Fig. 12- 13.
2. Undo the clamp screw on the seat receiver and rotate the battery holder to the back. You can now remove the battery from the top. Fig. 14.
3. Release the back part of the scooter by lifting the interlock and push the front part to the side. Fig. 15 - 16.
4. Pull and rotate the locking knob to fix the castors for straight running. Fig. 17.
5. Remove the basket.
6. Undo the adjustment knob of the steering column.
7. Fold the steering column down and retighten the adjustment knob. Fig. 18.

When assembling the scooter follow the steps in reverse order.

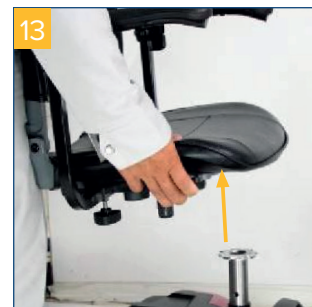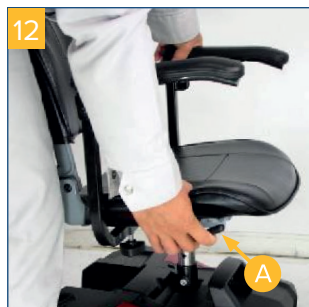Storage

Make sure that the scooter is switched off and is stored with the battery removed and the charger unplugged. Store the scooter in areas which are protected against direct sunlight, water or damp.

Cleaning | Disinfection

- Clean regularly with a damp cloth by hand!
- Do not use any aggressive detergents!
- Do not use any abrasive detergents!
- Do not use any high-dose chlorine or vinegar-based cleaners!
- For disinfection you can use a spray disinfectant, e.g. Descosept AF or Bacillol AF.
- Regularly monitor the screws and re-tighten them if required!
- The product must not be exposed to temperatures over 60° C for a sustained period.
- If there are defects, call the dealer from whom you bought this product.
- Do not use petrol, solutions or vaporising solutions as these could damage the body panels.
- Make sure that the charger has been removed and that the scooter is switched off before you clean it.
- Do not park your scooter in damp or misty conditions as this will lead to the formation of mould and rapid deterioration of the padded parts.

Troubleshooting

Problem	Cause	Remedy
You can't switch the scooter on	<ul style="list-style-type: none"> Batteries not connected Safety switch/overload switch or fuses have been activated 	<ul style="list-style-type: none"> Check battery connection and fuses Press the safety switch to reset
You can switch the scooter on, but it won't drive	<ul style="list-style-type: none"> Defective or weak batteries Charger is plugged in Motor is in freewheel 	<ul style="list-style-type: none"> Charge/replace battery Unplug the charger from the scooter Move the freewheel lever to "Drive"
Scooter drives slowly	<ul style="list-style-type: none"> Weak batteries Speed is set to low 	<ul style="list-style-type: none"> Charge battery Turn the speed selector up
The seat moves when in use	<ul style="list-style-type: none"> The seat is not properly clicked into place or is incorrectly fitted. 	<ul style="list-style-type: none"> Rotate the seat slowly until it clicks into place and is secure, refit if necessary (see "Seat" section).
The steering column seems to be loose or does not swivel freely	<ul style="list-style-type: none"> The fixing screws for the steering column are loose 	<ul style="list-style-type: none"> Tighten the fixing screws for the tiller
Horn sounds unintentionally or automatically	<ul style="list-style-type: none"> The scooter has diagnosed a fault 	<ul style="list-style-type: none"> Make sure that the drive lever is not activated and switch the scooter off and on. Charge the batteries if the problem continues.

Inspection and servicing

Checks

Your scooter has been designed so that it only needs minimal servicing. However, as with all motorised vehicles, regular maintenance work is required. So that you can enjoy many years with your scooter, we recommend that you follow the maintenance points on the plan below:

Weekly check

Check the following points each week or before driving. If you discover any irregularities, contact your Drive Medical dealer, in order to ask for advice or arrange an inspection.

Steering column

Make sure that it is not loose and that you can steer easily.

Speed controller

Make sure that this can be adjusted smoothly.

Drive lever

Make sure that the scooter drives when the drive lever is pushed and stops when the lever is released.

Motor

Be aware of unusual noises and make sure that the electromagnetic brake is working properly.

Freewheel lever

Make sure this is working properly.

Seat

Make sure that it can be rotated and locked in place easily.

Tyres

Check the profile depth and look for signs of damage, such as, for example when checking the tyre pressure (2.5-3.5 bar).

Monthly check

Carry out a visual inspection of the cable harness of the control unit. Make sure that these are not frayed or cut and that there are no loose threads.

Keep the control unit clean and protect it from rain or water. Keep the wheel hubs free of fluff, hair, sand and carpet fibres.

Carry out a visual inspection of the tyre profile. If it shows any signs of damage or wear, ask your dealer to come and replace the tyres.

Check the seat and backrest occasionally for sagging, cuts and tears. Replace as necessary.

All moving mechanisms can be serviced by simple lubrication and inspection.

Lubricate with grease or a light oil. Do not use too much oil, as small drops could cause staining and carpets, panels, etc. could be damaged. Always carry out a general check to make sure all nuts and screws are tight.

Six-monthly checks

Check the carbon brushes (motor brushes). We recommend that your dealer checks the brushes every six months or earlier, if your electric scooter is not working properly.

If the inspection shows excessive wear of the carbon brushes (motor brushes), then these must be replaced as otherwise the motor will be damaged.

The carbon brushes (motor brushes) should be checked for wear and the colour of the braid within the spring must be checked. If the braid is dark brown, red, silver, purple or gold, the carbon brush (motor brush) must be replaced.

Inspecting or replacing the motor brushes

Undo the end caps of the carbon motor brushes (by using a screwdriver on the cap).

Remove the carbon brushes (motor brushes).

Check the carbon brushes (motor brushes) for wear (replace if less than 6 mm).

Replace the carbon brushes (motor brushes) as required.

Check the condition of the battery connections every six months. Make sure that this is not corroded and that the connections are tight. Regularly apply a thin film of vaseline on the connection surfaces, to protect them from corrosion.

Annual checks

To make sure that your scooter is correctly maintained, have it checked by your Drive Medical dealer regularly. We recommend having the scooter serviced at least once a year. This service is not free of charge.

Guarantee

Any work or fitting of spare parts must be carried out by an authorised Drive Medical dealer/Customer Service representative.

To make sure the warranty applies, when your scooter re

quires customer service, please contact your Drive Medical dealer/Customer Service representative.

If a part of the scooter should require repair as a result of a manufacturing or material fault within 24 months of receipt of the scooter or the scooter must be replaced in full or in part, then replacement parts are delivered free of charge.

The repaired or replaced parts will be covered through this warranty for the warranty period for the scooter. Parts subject to wear are generally not covered during the normal period of warranty, unless items require repair or replacement and this is necessary as a clear direct result of a manufacturing or material fault.

Parts subject to wear (amongst others):

Pads, tyres, carbon brushes and batteries.

The warranty conditions given above apply for a new scooter, which has been purchased at the full sales price. If you are unsure whether your scooter is covered by the warranty, find out from your Customer Service representative.

Under normal conditions, no responsibility is accepted, if

the scooter is not working as a direct result of one of the following:

- The scooter or a part of the scooter was not serviced in accordance with the manufacturer's recommendations.
- Original parts have not been used.
- The scooter or a part has been damaged as a result of negligence, accident or incorrect use.

Please note that the contact details for your local Customer Service representative can be found in the box on the back page. In the event that your scooter requires customer service, please contact Customer Service and pass on all relevant details so that they can respond rapidly.

Technical specifications

Product name	CROW
Model	ST3
Max. Speed	6 km/h or 8 km/h; depending on version
Batteries	22Ah AMG; 30,15Ah Li-Ion; depending on version
Motor	250 W
Turning radius	145 cm
Climbing capacity	8° / 14%
Ground clearance	5.2 cm
Front tyres	225 mm / 9"
Rear tyres	225 mm / 9"
Total length	112 cm
Total width	59 cm
Total height	93.5 cm
Weight without/with battery	55 kg
Max. Load	130 kg

Serial number

In order to guarantee the proper Aftersales Service and Customer Service for warranty claims, please enter the serial number of the scooter. The serial number is located on the type plate or the front right of the frame in the wheel arch.

Serial number	
Motor serial no.	
Control unit serial no.	

Note the details of your Drive Medical dealer here:

Dealer	
Address	
Postcode	
Telephone	

Maintenance report

YEAR	1	2	3	4	5	YEAR	1	2	3	4	5
Maintenance dates						Maintenance dates					
Control unit						Padding					
On/off switch						Seat					
Control lever						Backrest					
Braking system						Armrests					
Charging connection						Electronics					
Batteries						Condition of the cable connection					
Capacities						Lights					
Connections						Test run					
Discharge test						Forwards					
Wheels and tyres						Backwards					
Wear						Emergency brake					
Pressure						Left rotation					
Bearing						Right rotation					
Wheel nuts						Test on a slope					
Motors						Over obstacles					
Wiring						List of repaired elements					
Noise											
Connections											
Brake											
Brushes											
Chassis											
Condition											
Steering system											

A detailed maintenance report can be downloaded from our homepage.

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Drive Medical GmbH & Co. KG
MEDNET EC REP GmbH,
Chien Ti Enterprise Co., LTD.