ENVOY

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Table of contents

Preface.....	17
Declaration of conformity.....	17
Service life	17
Disposal.....	17
Identification plate.....	17
Reuse.....	17
Warranty.....	17
Parts of the product	18
Indication.....	19
Scope of delivery.....	19
Safety instructions.....	19
Introduction	20
Purpose.....	20
Electromagnetic compatibility (EMC).....	20
Instrument panel	21
Use.....	21
Batteries	22
Tires.....	24
Driving in the street.....	24
Transport.....	25
Storage	25
Cleaning Disinfection	25
Troubleshooting	25
Inspection and maintenance	27
Warranty.....	28
Technical data.....	28
Serial number.....	28
Maintenance report.....	29

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the described product there is conformity according to the Regulation on medical devices (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

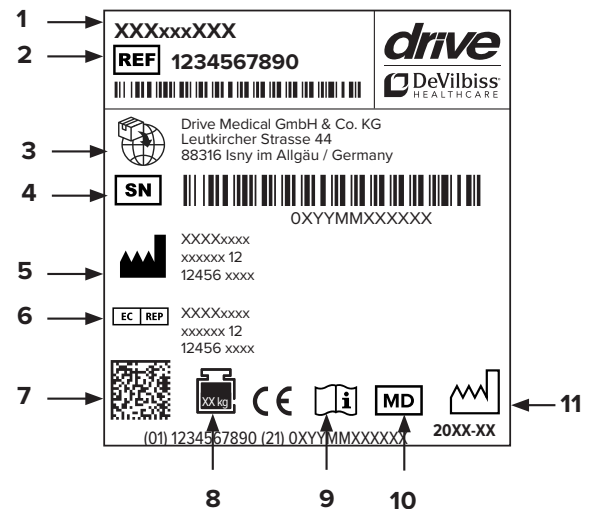1 Article name | 2 Article number | 3 Importer | 4 Serial number | 5 Manufacturer | 6 EC-REP | 7 UDI code | 8 Max. load | 9 Follow the instructions for use | 10 Medical device | 11 Production date (year/month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available at www.drivedevilbiss.de or from your specialist dealer.

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase and 6 months for the battery from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others):

Tires, handles, cushions, batteries, carbon brushes

Parts of the product

ENVOY

1. Seat
2. Armrest
3. Seat adjustment: Rotation
4. Side reflector
5. Front suspension
6. Headlight
7. Seat adjustment: Front/back
8. Steering column adjustment
9. Basket
10. Front indicator
11. Rear view mirror
12. Neutral lever
13. Height adjustment of armrest
14. Tail light
15. Rear indicator
16. Anti-tilt aid

Indication

INDICATION:

Significant impairment of mobility/walking in the case of structural and/or functional damage to the lower extremities (including amputation, after-effects of injuries, musculoskeletal/neuromusculoskeletal movement disorders)

- Preserved coordination function of the arms and hands to drive the vehicle
- Sufficient orientation and coordination ability
- Preserved residual ability to stand and walk (a few steps)
- To ensure mobility outdoors (getting to local areas and performing everyday activities)

When using outdoors/in street traffic, it should be ensured that proper operation of an electric motor drive is possible and that there are no concerns regarding fitness for driving.

CONTRAINDICATION:

Persons without sufficient functions regarding, among others, vision, hearing, consciousness, orientation, awareness, coordination of arm and hand movements should be excluded from using an electric scooter.

Scope of delivery

1 x	Electric scooter
1 x	Charging unit including instructions for use
1 x	Instructions for use
	Cane holder (optionally available)

Safety instructions

The scooter is a high-performance machine. Please read the following instructions carefully to protect yourself at all times.

General

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Never use the scooter if you are under the influence of drugs, alcohol, or certain medications which impair your safety.
- Never use electronic radio devices, such as walkie-talkies or mobile telephones.
- When moving in reverse with your scooter, ensure that there are no obstacles behind you.
- Do not make sudden directional changes at full speed, particularly on uneven or hilly surfaces. Failure to observe this can result in the scooter tipping over.
- On public roads, use your scooter on the bicycle path, if possible.
- Do not drive over curbs whose height exceeds the ground clearance of your scooter. Approach curbs slowly and at a slight angle while holding firmly onto the handlebars with both hands.
- When driving the scooter, always keep both hands on the handlebars and your legs in the scooter.
- Do not drive your scooter in the snow in order to avoid accidents on slippery roads.
- Only change the direction of travel (forward, reverse) while stationary.
- Do not drive any passengers and do not exceed the total weight. (Always observe the technical specifications for your model!).
- Never switch on your scooter while pressing the forward/reverse lever.
- Never get on or off your scooter when the scooter is not switched off.
- Always switch the scooter "off" before you swivel the seat.
- In confined spaces, drive only at the lowest speed setting.
- Ensure that there is sufficient space to turn in narrow alleys and corridors. Because of the way they are constructed, there may be problems when turning the scooter.
- Only drive your scooter when the seat is snapped into the driving position.
- Do not drive your scooter over a deep, soft surface (such as soft sand, thick grass, loose gravel).
- Do not drive your scooter on slopes which exceed 16%.
- Always proceed carefully when driving your scooter, especially when you are approaching a slope.
- Always proceed carefully when you are driving on uneven surfaces.
- NEVER drive your scooter through deeper water and never clean your scooter with a high-pressure cleaner.
- Never expose the scooter for a longer period of time to direct sunlight or other sources of heat. The seat surface and metal parts may present a risk of burns.
- Never expose yourself and the scooter to extreme cold for a longer period of time. Injury to the user or damage to the electric scooter can otherwise not be excluded.

- Do not allow children to play unsupervised near this device while the batteries are charged.
- Do not exceed the maximum load limits of your vehicle.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.

Battery

- Do not drop, strike, crush, or otherwise abuse the battery since the corrosive battery cell contents can be exposed as a result.
- If the battery leaks, do not allow the liquid to come into contact with skin or eyes. In the case of contact, wash the affected area with plenty of water and seek medical advice.
- Do not expose the battery to fire or extreme heat. Do not burn. Do not expose the battery to intense heat since this can cause an explosion. Avoid storage in direct sunlight.
- Do not expose the battery to water, rain, or any type of moisture.
- Do not crush, disassemble, pierce, or short-circuit the connection terminals.
- Do not open, disassemble, or try to repair the battery; it does not contain any parts which can be repaired.
- Do not arbitrarily store the batteries in a container in which they can short-circuit each other or be short-circuited by other metal objects.
- Keep batteries out of the reach of children.
- Keep batteries clean and dry.
- Use the battery only for the application for which it is intended.
- Regularly check the connection cable, connection heads, and power supply for a secure fit, damage, or signs of wear. Do not use if damaged.
- Fully charge the battery prior to initial use.
- Max. time between charges = 6 months.
- Recommendation: Store the battery at below 25°C, with low air humidity, and in a dust-free and non-corrosive gas atmosphere. Whenever possible, store batteries fully charged.
- This device contains electrical and/or electronic components. Be aware of local regulatory and recycling regulations for disposing of the device components.
- Dispose of the battery properly.

Introduction

Please read this user manual carefully before operating the device. The improper use of the electric scooter can lead to damage, injuries, or traffic accidents.

This manual explains how you can optimally use your electric scooter. It includes instructions for the operation, assembly, and maintenance of the electric scooter.

This manual also contains a repair and maintenance table and warranty information. Please keep the manual in a safe place or directly with the electric scooter.

If other people are to use the electric scooter, make sure to give them the manual to review.

Since the design is subject to changes, some of the illustrations and photos in the manual may not match the electric scooter you purchased. We reserve the right to make design changes.

The electric scooter was constructed and built to offer a comfortable, safe, and simultaneously affordable solution for certain mobility requirements.

Purpose

The electric scooter is designed for outdoor use for persons with limited walking ability. It is thus able to overcome distances and obstacles outside. It is classified according to DIN EN 12184 in category B.

Electromagnetic compatibility (EMC)

This product was tested for electromagnetic compatibility (EMC) according to IEC 60601-1-2 and ISO 7176-21.

Instrument panel

1. Battery display
2. Speed controller
3. Warning light
4. Horn
5. Indicators
6. Light switch - headlight
7. Control lever
8. Fast (H) / slow (L) drive switch

Use

Ignition key

To start the electric scooter, turn the key to the right; the instrument panel is now lit up. To switch off, turn the key to the left; the instrument panel turns off and the key can be removed.

Warning: Never switch off the ignition while you are driving since this leads to an immediate stop and possible damage or injury.

Control levers

The electric scooter is driven by operating the control lever on the side of the instrument panel. To drive the electric scooter forwards, move the right control lever in the direction of the operator. To drive the electric scooter backwards, move the left control lever in the direction of the operator. The horn generates a beep when the vehicle moves backwards. The longer the control lever is pressed, the faster the electric scooter moves.

To stop the electric scooter, release the control lever; the electromagnetic brake automatically engages.

Warning: Never operate the left and right control levers at the same time.

Brakes

By releasing the control lever, the vehicle brakes automatically. When the vehicle comes to a stop, the magnetic brake automatically closes. The scooter is now secured against rolling away.

In an emergency, the user can stop the electric scooter by operating the hand brake on the left on the hand grip of the steering column.

Warning: Never put the vehicle into neutral on inclines. In this case, the brake will not grip.

Setting the speed

The speed of the electric scooter can be increased by turning the speed controller to the right (rabbit symbol) or reduced by turning the speed controller to the left (turtle symbol).

Warning: Never adjust the speed setting while driving since this can lead to a loss of control. Do not select the maximum speed when driving indoors.

When the electric scooter is used indoors or in pedestrian areas (such as sidewalks or shopping areas), the lowest speed setting should be selected. If the electric scooter is used on the street, the high speed setting can be selected.

Press the “H/L” button (pic. 1) to switch between the two speed levels. In the slow speed level, the speed is reduced by about half.

Warning: Always use the low speed setting when you are driving indoors or on a sidewalk.

Note: Set the speed to maximum to drive up an incline and to minimum for driving downhill.

Horn

By pressing the “horn” button, a beep sounds; the beep is muted as soon as you release the button.

Indicators

Use the respective indicator to indicate your direction when driving around a curve. Press the same button again to switch off the indicator.

Light

Press the light button once to switch on the headlights. Press the light button again to switch off the headlights.

Warning light

Press the warning light button once to switch on the warning light system. Press the button again to switch off the function. When the button is pressed, the warning light function causes all indicators to blink and the horn sounds.

Seat

The seat can be rotated 360° and snaps into place in 45° increments. To rotate the seat, pull the lever (A) under the left side of the seat upwards and then rotate the seat. Release the lever and continue rotating the seat until it snaps into place.

You can adjust the space for your feet on the electric scooter by pushing the seat back and forth. To shift the seat, pull the handle for shifting the seat (B) upwards. Then place the seat in the proper position, release the handle, and continue pushing the seat until it snaps into place.

Removing the seat: Pull lever A upwards and rotate the seat approx. 10° to the left and lift it up.


Mounting the seat: Before putting the seat on again, pull lever A upwards once again and put the seat fully on. Then release lever A. After correct mounting, the seat must once again snap into place in 45° increments.

Important: You must hear and feel the seat snap into place.

Caution: Bring the seat back into the forward position before you drive.

Adjusting the armrest

First, unscrew the star screws at the back on the seat frame (A), then adjust the distance between the armrests by moving them. Once the correct distance has been adjusted, secure the screws once again. To adjust the heights of the armrests, unscrew the star screws directly on the armrests (B) and adjust the height by moving the armrests. Retighten the screws.

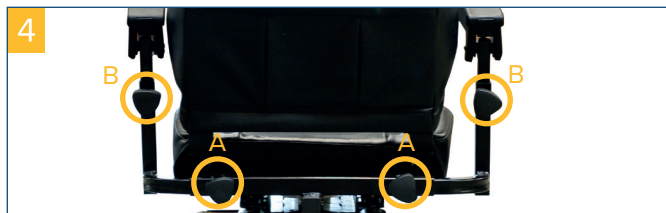Adjusting the steering column

The steering column can be adjusted in various positions:

- Unscrew the screw to adjust the steering column in the desired position. Retighten the screw.

Neutral lever

The neutral lever enables a release of the automatic brake so that the electric scooter can be pushed and/or maneuvered by hand.

To allow the scooter to be moved, the scooter should be switched off.

In neutral mode the brakes do not grip, therefore it should not be used if:

1. Someone is sitting in the electric scooter
2. The electric scooter is left unattended
3. The electric scooter is on a slope

Driving mode/neutral mode

Driving mode:

Move the lever to **Drive**: The electric scooter is driven by a motor.

Neutral mode:

Move the lever to **Neutral**: The electric scooter is in neutral mode and can now be pushed/moved by hand.

Batteries


Battery display

The battery charge status can be read on this display.

Green shows a remaining charge of (40~100%)
Yellow shows a discharging of (10~30%)

Red shows that the battery must be recharged as soon as possible.

The remaining charge indicated by the battery display depends on the driving time and your driving style. Repeated starting and stopping and traveling uphill use up the remaining energy more quickly.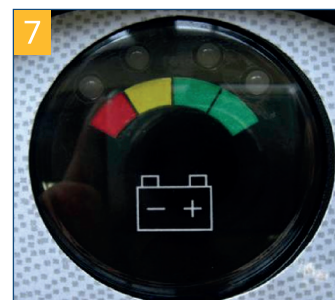

Charging the battery

Read the enclosed instructions for use of the charging unit. If the scooter is not used for a longer period of time, ensure that the batteries are charged at least once per month to avoid deterioration of the condition of the battery.


Changing the batteries

It is normal for the battery capacity to decrease over time, even if the battery is always charged as described above. When the battery performance reaches only 60% of its former capacity, we recommend changing the battery. Continued use of the old batteries leads to a rapid reduction of the range of the electric scooter and can lead to significant wear of other electric scooter parts. It is recommended to replace the batteries in pairs and to obtain them from your dealer.

Installation and removal of the batteries

1. Switch off the electric scooter at the ignition and remove the seat.
2. Unscrew the two fixing screws of the battery compartment cover.
3. Remove the battery compartment cover and disconnect the connection for the tail light and indicator lights.
4. Disconnect the battery connections.
5. Remove the Velcro strap from the batteries and take the batteries out.
6. Installation in the reverse order.

Correct connection of the batteries:

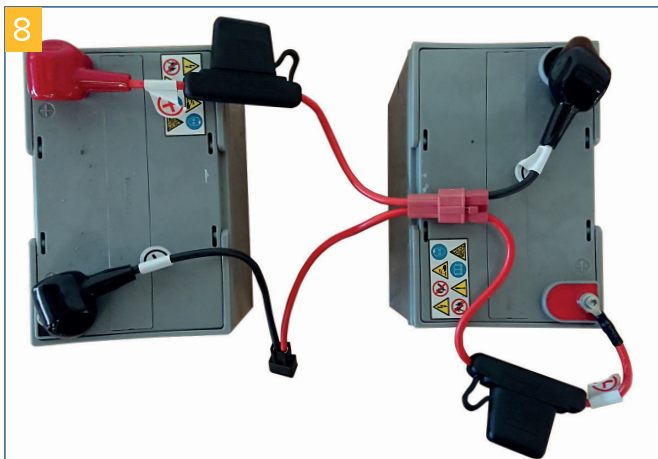Battery care

You should charge the batteries after every time you use the electric scooter in order to ensure the maximum battery range. The batteries should be charged at least once per month, even if the electric scooter is not being used.

After charging or inserting a new battery, drive the electric scooter a short distance to ensure that the battery capacity is sufficient.

In cold environments, the battery may react more slowly and the range is shortened accordingly. When traveling on inclines and slopes, the battery display fluctuates. This is a normal process.

The battery range is reduced when driving over inclines or uneven surfaces since the electric scooter needs more power.

The batteries should not be charged for more than 24 hours. The batteries are maintenance-free and should accordingly not be filled with water. The batteries should be regularly checked to ensure they are clean. For information about this, please read the section of this manual below.

Cleaning the battery

If the batteries become contaminated with water, battery acid, dust, or other substances, they discharge faster. The batteries provided with the electric scooter are sealed and as such, they are maintenance-free and leak-proof. Follow the steps below to clean the battery.

Ensure that the electric scooter is switched off. Remove the seat and the battery cover. Use a clean cloth to clean the soiled surfaces. Take the battery out.

Clean the battery with a clean cloth. If the terminals are covered with a white powder, wipe them using some warm water.

Warning:

The cables for the electrical system, charging unit, and batteries are already properly laid in the factory. Do not attempt to lay the cables yourself. Improperly laying the cables can cause the cables to become pinched in the battery compartment which in turn can lead to a failure of the electrical system. Ensure that the battery cables are connected at the correct battery poles. A connection diagram can be found underneath the battery compartment cover.

Recommendations:

If necessary, please ask your dealer for assistance with maintenance and replacing the battery.

1. Ensure that the battery cables are connected properly and secure the cover once again.
2. Do not use the battery to charge telecommunication devices or other articles.
3. The battery performance varies with outdoor conditions; the driving performance is lower in the winter. If the vehicle is not driven for a longer period of time, charge the battery at least once per month.
4. Replace both batteries at the same time.

Overload switch

Because of excessive loads, for example, due to steep inclines, the overload switch can pop out in order to protect the motor and electronics from damage.

**Tires**

The condition of the tires is influenced by how you drive the electric scooter and on what surface it is driven. Please check the tread depth of the tires regularly. Replace tires when the tread depth is less than 1 mm. The tire pressure for the best handling and range should be approx. 2.5 - 3.5 bar.

Driving in the street

You may drive your electric scooter in traffic on the sidewalk or bicycle path. Please be aware of the following: You must ensure in any case that your scooter is in proper working condition.

This includes:

- A functional braking system.
- Passive illumination: Red reflectors in the rear and yellow or white reflectors on the sides.
- Active illumination: White headlights in the front, red rear lights in the rear, white or yellow blinkers as directional indicators.

Since your speed in street traffic could be underestimated or since you may be overlooked by other people in traffic, you should take preventive steps:

- Wear light, reflective clothing and wear your safety belt (if available).
- You may wish to attach a signal flag on a flexible post to the scooter. These flags are available in a bicycle accessory shop.
- Always drive defensively and anticipate the behavior of other people on the road.
- Check whether the seat is correctly installed and snapped into place in the forward position.
- Ensure that the steering column and the seat are oriented for a comfortable driving position.
- Fold the armrests down so that you can place your arms on them.
- Switch the ignition on. If necessary, switch the headlight on.
- Check the battery display to determine whether the current is adequate for your trip. If you are not sure regarding the remaining energy, charge the batteries before your trip.
- Set the speed selection to a setting which is safe and comfortable for you.
- Check that the control lever is working properly.
- Check that the electromagnetic brake is working properly.
- Ensure that your immediate environment is safe before you drive on the road. Use a low speed setting when driving on a sidewalk or indoors.
- **Caution: Never operate the left and right side of the control lever at the same time. As a result, you will not be able to control your electric scooter and this can lead to damage or injuries.**
- Never switch off the ignition while you are driving since this leads to an immediate stop and possible damage or injury.
- Do not select the maximum speed when driving indoors.
- Do not switch speeds while driving. A sudden change in speed can jeopardize you and others and damage your electric scooter.
- Do not place any magnetic objects near the electrical parts of the electric scooter since these could impair the safe operation of the electric scooter.
- Use caution when driving in high-traffic or crowded areas.
- When driving in reverse, be aware of people and objects behind your vehicle. During reverse driving, the electric scooter emits an acoustic warning sound.

Stopping and parking

Completely release the control lever. The vehicle brakes automatically and stops.

Switch the ignition key to OFF. Then take the key out.

Transport

- Use a ramp to load the scooter. If necessary, you can also lift the scooter over a step or the like. At least two people are needed to do this. Lift the scooter only at fixed parts of the frame, in the front and rear under the cladding. For safety reasons, the scooter may be lifted only WITHOUT the user.
- Do NOT remain seated in your scooter when the scooter is being transported in a vehicle. Always secure your scooter and use a seat in the vehicle.

Exception: Taking your scooter with you in public transport buses.

Storage

Ensure that the electric scooter is switched off and stored with the battery removed and the charging unit unplugged. Store the electric scooter in areas protected from direct sunlight, water, and humidity.

Cleaning | Disinfection

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product must not be continuously exposed to temperatures over 60°C.
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Do not use any petroleum ether, solutions or evaporating solutions since these could damage body parts.
- Ensure that the charging unit was removed and that the electric scooter is switched off before you clean it.
- Do not park your electric scooter under wet or damp conditions since this leads to mold growth and a rapid deterioration of the cushioned parts.

Troubleshooting

Problem	Cause	Remedy
Electric scooter cannot be switched on	Batteries not connected Circuit breaker/overload switch was triggered	Check battery connection Press circuit breaker to reset
Electric scooter can be switched on but does not drive	Defective batteries Charging unit is inserted Motor is in neutral	Charge/replace battery Remove charging unit from the electric scooter Put neutral lever into "Drive"
Electric scooter drives slowly	Weak batteries Speed is set to low	Charge battery Select higher speed
The seat moves during operation	The seat is not properly snapped into place	Rotate the seat slowly until it snaps into place and is secured (see section entitled "Seat").
The steering column appears to be loose or does not freely swivel	The fixing screws for the steering column are loose	Tighten the fixing screws for the steering column
Horn sounds unintentionally or automatically	The electric scooter has diagnosed an error	Ensure that the control lever is not operated and switch the electric scooter off and on. Charge the batteries if the problem persists.

Error code description S-Drive

If the red LED in the battery display is blinking, the error can be determined using the blink code:

Number of blinks	Error	Effect on the scooter	Comments
1	Low battery voltage	Driving is not possible or slow	The charge status of the batteries is low. Charge the batteries immediately. If there is no charging, check the battery cables to ensure that they are positioned securely and connected properly.
2	Motor not connected	Driving is not possible	The contact between the motor and the control unit is poor; check all plug connections from and to the motor.
3	Motor wiring problem	Driving is not possible	The motor or its cable is defective. Check exposed cables or short-circuited electrical circuits. Contact your customer service.
4	Freewheel activated	Driving is not possible	Check whether the gearshift lever for the parking brake is in the locked position. The brake coil or the cables of the parking brake are defective. Check the parking brake and the cables for short-circuited electrical circuits or exposed cables.
5	not allocated		
6	Lock active	Driving is not possible	The charging unit is still connected. Please disconnect the charging unit from the vehicle.
7	Error of the rotational speed potentiometer	Driving is not possible	The throttle lever or its cable is defective. Check exposed cables or short-circuited electrical circuits. The throttle lever could be incorrectly adjusted. Contact your customer service.
8	Control unit defective	Driving is not possible	Check the plug connections on the control unit; possibly replace the control unit.
9	Parking brake	Driving is not possible	The brake coil or the cables of the parking brake are defective. Check the parking brake and the cables for short-circuited electrical circuits or exposed cables.
10	Battery voltage is too high	Driving is not possible	The charge status of the batteries is too high. Pull out the charging unit connection if it is connected or rotate the "Charge/Run" switch into the "Run" position. Scooters which are operated with RHINO charge batteries while the scooter rolls down an inclined surface or decelerates. This error can be caused by excessive charging in this manner. Switch the scooter off and then back on.

Inspection and maintenance

Inspections

Your electric scooter was developed such that it requires only minimal maintenance. However, like all motorized vehicles, it requires regular maintenance work. In order for you to enjoy your electric scooter for many years, we recommend observing the following maintenance points according to the plan:

Weekly inspection

Check the following points every week or prior to driving. If you notice an abnormality, contact your Drive Medical dealer for advice or to schedule an inspection.

Steering column

Ensure that this is not loose and that it can be easily controlled.

Speed controller

Ensure that this can be smoothly adjusted.

Control levers

Ensure that the electric scooter drives when the lever is pressed and stops when the lever is released.

Motor

Check for unusual noises and that the electromagnetic brake is operating correctly.

Neutral lever

Ensure that it is working properly.

Seat

Ensure that it can be easily rotated and snaps into place.

Tires

Check the tread depth and take note of signs of damage, such as checking tire pressure (2.5-3.5 bar).

Monthly inspection

Perform a visual inspection of the cable harness of the control system. Ensure that this is not frayed or cut and that no wires are exposed.

Keep the control system clean and protect it from rain or water. Keep the hubs free of lint, hairs, sand, and carpet fibers.

Perform a visual inspection of the tire tread. If this demonstrates damage or wear, have the tires changed by your dealer on site.

Check the seat and backrest occasionally for sagging, cuts, and tears. Replace if needed.

All moving mechanisms are maintained through simple lubrication and inspection.

Apply grease or a light oil. Do not use too much oil, since small drops can lead to staining and damage carpets, clothing, etc. Always generally check whether all nuts and screws are tightened.

Semiannual inspection

Check the carbon brushes (motor brushes). We recommend that your dealer check all brushes every six months or less if your electric scooter does not operate smoothly.

If the inspection reveals excessive wear of the carbon brushes (motor brushes), these must be replaced since otherwise the motor will be damaged.

The carbon brushes (motor brushes) should be checked for wear and the color of the mesh inside the spring. If the mesh is dark brown, red, silver, purple or golden, the carbon brush (motor brush) must be replaced.

Inspection or replacement of the motor brushes

Loosen the caps of the carbon brushes (motor brushes) (by using a screwdriver on the caps).

Remove the carbon brushes (motor brushes).

Check the carbon brushes (motor brushes) for wear (exchange if less than 6 mm).

Replace the carbon brushes (motor brushes) if needed.

Check the condition of the battery connections every six months. Ensure that they are not corroded and that the connections are secure. Regularly apply a thin film of Vaseline to the connection surfaces to protect them from corrosion.

Annual inspections

To ensure that your electric scooter is properly maintained, have it inspected regularly by your Drive Medical dealer. We recommend having the electric scooter maintained at least once annually. This service is not free of charge.

Warranty

Any work on or installation of replacement parts must be performed by an authorized Drive Medical dealer/customer service employee.

In order for the warranty to apply, please contact your Drive Medical dealer/customer service employee if your scooter requires customer service.

If a part of the scooter should require repair or the scooter needs to be replaced in whole or in part due to a manufacturing or material error within 24 months of receipt of the scooter, replacement parts will then be delivered free of charge.

The repaired or replaced parts are covered by this warranty for the warranty period of the scooter. Parts subject to wear are generally not covered during the normal warranty period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

Parts subject to wear include (among others): Cushions, tires, motor brushes and batteries.

The above warranty conditions apply to a new scooter which was purchased for the full sales price. If you are unsure whether your scooter is covered by the warranty, please inquire with the customer service employee. Under normal circumstances, we do not accept responsibility if the scooter does not work as a direct consequence of the following:

The scooter or a part of it was not maintained according to the manufacturer's recommendations.
Original parts were not used.

The scooter or a part was damaged as a result of negligence, an accident, or improper use.

Please note that the contact information of your local customer service employee are in the field on the back. If your scooter requires customer service, contact the local customer service employee and provide all relevant details so that he/she can respond quickly.

Technical data

PU tires	without battery	36 Ah batteries	50 Ah batteries
anthracite	950200150-0	950200150-36	950200150-50
HMV no.	18.51.05.1028		
Tires	without battery	36 Ah batteries	50 Ah batteries
red	950200200-0	950200200-36	950200200-50
anthracite	950200100-0	950200100-36	950200100-50
HMV no.	18.51.05.1028		

Technical data	ENVOY
Speed	6 km/h
Battery capacity 2x	36 Ah / 50 Ah
Motor output	400 W
Turning radius	135 cm
Climbing ability	9° / 16 %
Ground clearance	8 cm
Tires	10" (260 mm)
Dimensions (LxWxH)	119 x 58x 117 - 122 cm
Seat width x Seat depth	48 x 45 cm
Weight without battery	76 kg
Max. load	160 kg

Serial number

To ensure proper after-sales service and customer service for warranty claims, please enter the serial number of the electric scooter. The serial number is found on the identification plate or on the back left on the frame.

Serial number	
Motor S/no.	
Control system S/no.	

Maintenance report

YEAR	1	2	3	4	5	YEAR	1	2	3	4	5
Maintenance dates						Maintenance dates					
Control system						Cushioning					
On/off switch						Seat					
Control lever						Backrest					
Braking system						Armrests					
Charge connector						Electronics					
Batteries						Condition of the cable connection					
Capacities						Lights					
Connections						Test run					
Discharge test						Forward					
Wheels and tires						Reverse					
Wear and tear						Emergency brake					
Pressure						Left turn					
Bearing						Right turn					
Wheel nuts						Test on a slope					
Motors						Over obstacles					
Cabling						List of repaired elements					
Noise											
Connections											
Brake											
Brushes											
Chassis											
Condition											
Steering											

This model meets the criteria and technical minimum requirements for e-scooters/electric scooters for roadworthy transport in local public transport buses according to the ordinance "Public bus transport as well as regular public transport services with coaches according to sections 42, 43 Public Transport Act (PBefG) in local public transport; obligation to carry e-scooters with seated person".

Thus transport in the intended local public transport buses and the mounting of the pictogram is possible.

Drive Medical GmbH & Co. KG, reserves the right to make changes to the draft and the specifications without prior announcement.

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