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Remote control support for *Instafold* Powerchair

Installation instructions

 Required tool:
Phillips screwdriver PH2 imprint

Contents of the bag:

- o 3 butterfly screws
- o 2 clamps
- o 1 joystick positioning support
- o 2 countersunk head screws
- o 2 washers

- 1** Unscrew the clamp from under the armrest (A), then remove the joystick and support rod.

- 2** Using the screwdriver, remove the 2 screws securing the joystick to the support rod (screws not to be reused in the following point 3).

- 3** Reverse the direction of the support rod so that this points towards the front of the joystick. Secure it using the 2 butterfly screws provided in the new kit.

- 4** Position the 2 clamps to point towards you, on the top foam covered bar of the powerchair, to a maximum of 12 cm from the edge. (Either right or left mounting)

- 5** Attach the positioning support and tighten the clamps using the 2 countersunk head screws and 2 washers supplied.

- 6** Insert the support rod and lock the position using the 3rd butterfly screw (B). Reposition the cable using the original fixings.

Proceed in reverse to return the joystick to the position armrest level. (note: during disassembly, compression marks may remain on the foam)

ME54079200