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Assembly Instruction Crutch holder

Product: Nitro Glide

Item code: 700500100

EN

The crutch holder for your Nitro Glide Knee Walker should be attached to the underside of the knee pad. For easy assembling, please read and follow the instructions below.

The assembling of the crutch holder should be done to meet individual patient requirements and can be fitted on the left or right of the unit, depending on the patients injured, non-weight bearing leg.

Step 1

Remove the knee pad from the box and remove all packaging material, or remove the knee pad from the Nitro Glide Walker.


Place the knee pad on its padded surface so that the stem tube faces upwards, as shown below. (pic. 1).

Step 2

Remove the four screws by turning anti-clockwise with an appropriate hexagon/Allen key (size 5/32) (Pic. 2).

Place the crutch holder brackets provided into position and replace the four screws. (Pic. 3).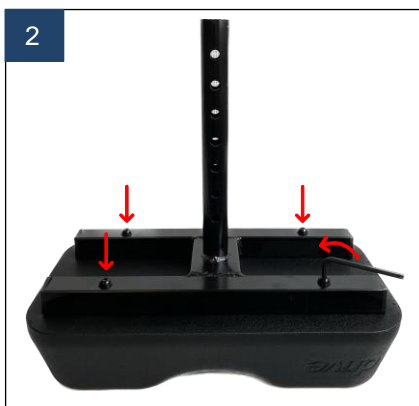

Assembly Instruction Crutch holder

Product: Nitro Glide

Item code: 700500100

Step 3

Now that the crutch holder brackets have been assembled, please refer to the Instruction for Use manual and install the knee pad.

To use the crutch holder, place one crutch top and bottom into the bracket clips as shown below. (Pic. 5).