GRAND XL

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Table of contents

Preface	7
Declaration of conformity	7
Service life	7
Disposal	7
Identification plate	7
Reuse	7
Warranty	7
Parts of the product	8
Intended purpose	8
Indication	8
Scope of delivery	8
Safety instructions	8
Unfolding	9
Height adjustment of the feet	9
Walking	9
Folding	9
Cleaning Maintenance	9
Technical data	9

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVEMEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

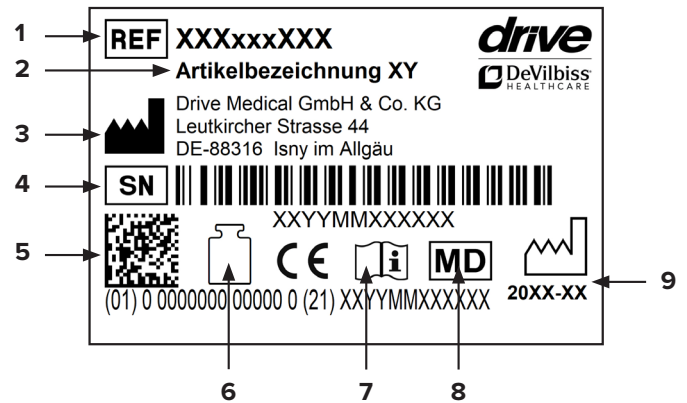
This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others):
rubber feet, handle

Parts of the product

GRAND XL

1. Handle
2. Locking mechanism
3. Adjustable foot
4. Rubber foot

Intended purpose

Walkers are used to support safe walking and, depending on the design, can also be used to relieve pressure. On some products there are devices or baskets to hold loads and objects, which can be transported with these frames. These walking frames can be folded for transportation and the handgrips are height-adjustable to suit different heights.

Indication

INDICATION:

Limitation of mobility in the case of impaired movement/coordination/balance – but still adequately preserved ability to walk if the use of other walking aids is not sufficient. To maintain/promote/safeguard walking and standing (gait training).

CONTRAINDICATION:

The product is not suitable for use in the case of: perception disorders, severe loss of balance, inability to stand freely.

Scope of delivery

1 x	Walking frame
1 x	Instructions for use

Safety instructions

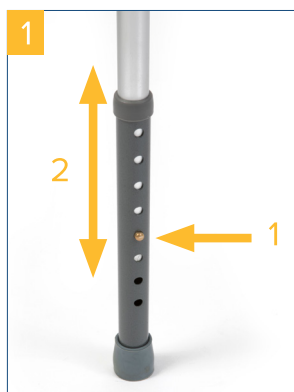
- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Use the product only as a walking aid!
- Use the product only on an even and solid surface.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Use on escalators and the like is expressly prohibited!

- Do not hang any bags on the handle – risk of tipping!
- When folding/unfolding the product, clothing and body parts can be pinched.
- Protect the product from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Prior to each use, ensure that your product is intact and all adjustable parts are properly secured.
- Check the screws on the product regularly and tighten, if necessary.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Unfolding

Place the folded walking aid in front of your body. Unfold the walking aid at the handles until the locking mechanism on the telescopic bar clicks into place. Check the locking mechanism.

Height adjustment of the feet

Before using the product, adjust it to the desired height. To adjust the height of the legs:

Press button (1) and at the same time, pull the leg out or push it in until the button clicks into place in the desired position (2).

Ensure that the pushbutton has correctly clicked into place. Ensure that all four legs are adjusted to the same height.

Walking

When walking, do not place the walking aid too far out in front. Always place the walking aid down on all four feet.

Take only small steps when using the walking aid and hold on to the handles. Stand firmly when moving the walking aid.

When diagonal to the direction of movement, the walking aid tips at an angle of more than 3.5°.

Only walk if the walking frame is stable.

Folding

Press down the locking mechanism and fold the side frames inwards.

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

Technical data

Product	Grand XL
Article number	10220HD
Overall length	52,5 cm
Overall width	62 cm
Overall height	82 - 100 cm
Dimensions when folded	62 x 19 x 82 cm
Weight	3,5 kg
Max. load	228 kg
Frame material	Aluminium

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Drive Medical GmbH & Co. KG