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DE

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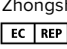
  
Com-Da Healthcare Co. Ltd.  
No. 1, Jinzhao Street, West District,  
Zhongshan City, Guangdong, CN  
  
Osmunda Medical Technology  
Service GmbH, Treskowallee 108,  
10318 Berlin, DE

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# INSTRUCTIONS FOR USE

EN

**⚠ WARNING:** To minimise the risk of injury from improper and unsafe use of this cane, read these instructions and safety precautions before using.

## Intended Purpose:

This cane is a walking aid to assist those who have reduced mobility.


## Safety Precautions:

- Consult a healthcare professional before using any assisted mobility device.
- Not to be used by persons heavier than 159 kg (25 st).
- Not intended to support the full weight of the user.
- Not intended for use on ice or other slippery surfaces as it has the potential to slip.
- Do not use unless individual segments are joined together securely, and the push button is fully engaged.
- When assembling individual segments, keep fingers away to avoid being pinched.
- Do not stretch the internal elastic cord more than necessary to assemble or disassemble the segments. Do not use the cane if the elastic cord is cut, frayed, or damaged.
- Applying downward pressure on the base (e.g. stepping on the base) while lifting the handle can cause the cane segments to separate and the cane to collapse, possibly causing you to fall.
- If the handle or pivoting base becomes loose, or if the cane becomes damaged, stop using the cane immediately.


**Indication:** The cane is suitable for use in the case of: walking impairments due to limitation of the active or passive musculo-skeletal system; coordination difficulties during walking and reduced resilience.

**Contraindication:** The cane is not suitable for use in the case of: perception disorders, severe loss of balance, inability to stand, user weight of above 159 kg

## SETUP

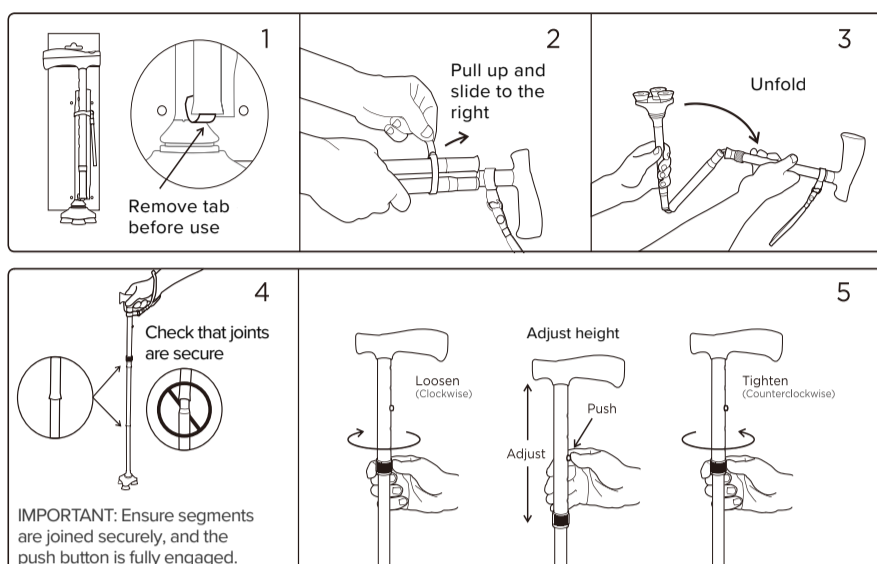## SAFETY PRECAUTIONS
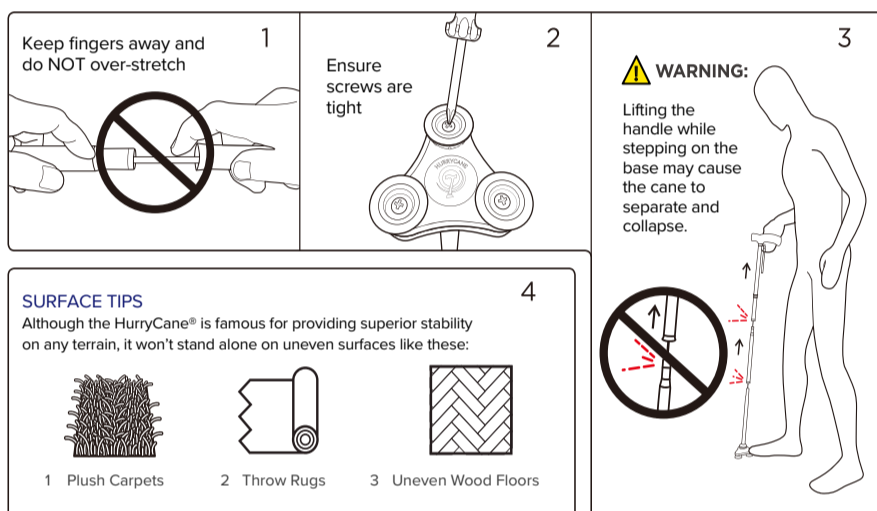
## MAINTENANCE

- Check the elastic cord before every use. Do not use the cane if the cord is cut, frayed, or damaged.
- Check the handle and pivoting base are secure.
- Periodically check the screws on the feet to be sure they are tight.
- Replace the feet every six months. Contact your local distributor to purchase new feet.
- Clean the cane occasionally using water and a mild detergent.

### SPECIFICATIONS

- HurryCane® Freedom Edition™
- Eight adjustable heights:  
From 77.5 cm to 95 cm  
(30 ½" to 37 ½")
- Folded size: 35 cm (13 ¾")
- Maximum user mass: 159 kg (25 st)


### Comments, Feedback, or Questions

For assistance with using your HurryCane® or for any warranty, sales or customer service information, please contact your distributor or retailer. If in doubt, contact Drive DeVilbiss Healthcare Ltd.

If a serious incident occurs in relation to the product, reports should be forwarded to the manufacturer and local competent authority.


## RESET THE FLEX

The pivoting base of a new HurryCane® is designed to have less flexibility initially, but will adapt to the user's movement over time. As the cane adapts to your movements, your HurryCane® may need to have its flex reset when you want the cane to stand beside you without holding on to it. Simply 'reset the flex' by following these 3 easy steps: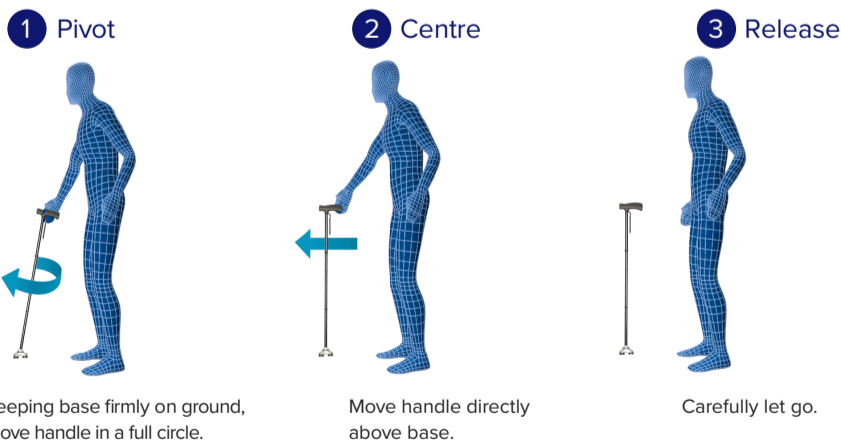

NOTE: If the cane falls over, you can raise it easily by putting your foot on the base and pressing down.

## STORAGE INSTRUCTIONS

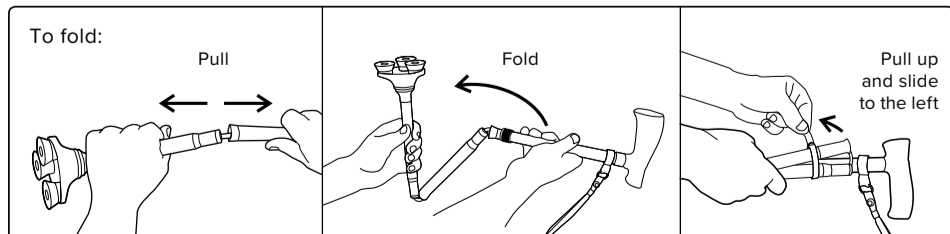When the product has come to the end of its useful life, follow local recycling and disposal policies.

Com-Da Healthcare Co. Ltd.  
No. 1, Jinzhao Street, West District,  
Zhongshan City, Guangdong, CN

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Drive DeVilbiss Healthcare Ltd,  
Sidhil Business Park, Holmfild,  
Halifax, West Yorkshire,  
HX2 9TN, GB  
+44 (0) 845 0600 333  
info@drivedevilbiss.co.uk

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Drive Medical GmbH & Co. KG,