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WS 016

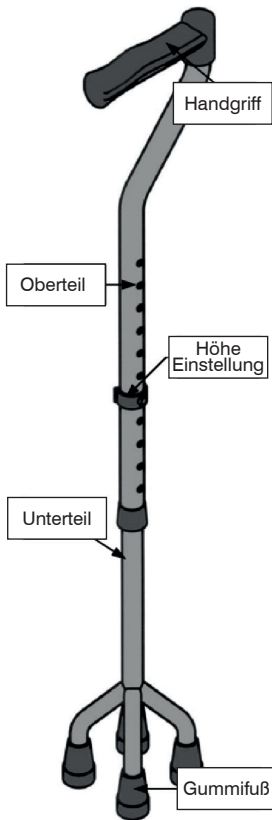
WS 016 L

**drive**

CE

[www.drivemedical.de](http://www.drivemedical.de)  
[www.drivemedical.co.uk](http://www.drivemedical.co.uk)

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
## About your Quad Cane

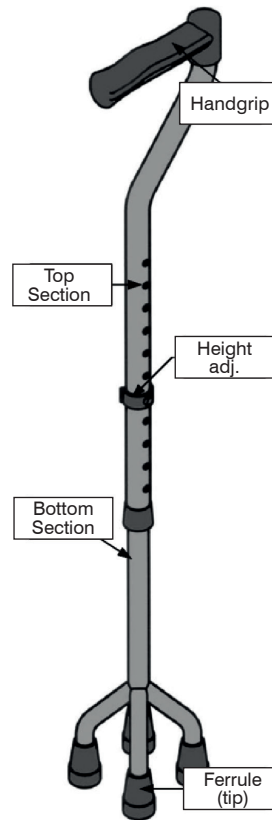WS016 Standard  
Model pictured

Your Drive Quad Cane is height adjustable, lightweight and suitable for indoor and outdoor use.

For extra security the bottom section has four legs, each tipped with a grey rubber ferrule.

Two models of Quad Cane are available:

The Standard model (**WS016**) has a height of 70 - 95 cm (27" -37"), it is suitable for users up to a user weight of 115 kg (18 stone).

The larger model (**WS016L**) has a height of 75 - 100 cm (30" - 39") and a larger footprint, it is suitable for users up to a user weight of 115 kg (18 stone) (differ from fig.).

The height is adjustable in steps of 25mm.

The distance between the feet on the outside is 115mm.

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## Adjusting the height

The ideal height for your your quad cane is the distance between the ground and the crease of your wrist. This should allow the user to have an upright posture with their elbow slightly flexed.

### For models with “E” clips

Pull out the “E” clip to remove. Move the top section until you reach the desired height then reinsert the “E” clip to secure.

### For models with brass buttons

Push in the brass buttons. Extend or retract the leg to the required height. When this is reached, release the button but carry on adjusting until the buttons lock in position.

## Unsafe, safety & maintenance

- Avoid wet floors, ice and similar hazards.
- Ensure there is sufficient grip on the surface you are using the cane on.
- Whenever possible use the stairs in the normal manner, using the handrail for support Use the Quad Cane on the same step as the affected leg and lead with the unaffected leg.
- Always ensure the top section is locked in position.
- Periodically check the rubber ferrules have not worn out. See your local Drive Medical dealer for replacement ferrules.
- Periodically check for worn or elongated adjustment holes.
- Always hold the Quad Cane by the handgrip during use.
- Do not exceed the maximum weight limit.

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The expected service life of the Quad Cane is 3 years for a single user if the instructions herein are followed. Failure to follow these instructions can reduce the service life of the device.

## Warranty details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to nondurable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stocklist.

**Drive Medical Ltd**

Ainley's Industrial Estate • Elland, West Yorkshire  
United Kingdom • HX5 9JP  
[www.drivemedical.co.uk](http://www.drivemedical.co.uk)

Date: 04.2021 | Subject to error and change

# drive

[www.drivemedical.de](http://www.drivemedical.de)  
[www.drivemedical.co.uk](http://www.drivemedical.co.uk)

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**Drive Medical GmbH & Co. KG**