NIMBO

Inhalt

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Table of contents

Preface	9
Declaration of conformity	9
Service life	9
Disposal	9
Identification plate	9
Reuse	9
Warranty	9
Parts of the product	10
Intended purpose	10
Indication Contraindication	10
Scope of delivery	10
Safety instructions	11
Preparing for use	11
Front wheel lock	11
Enable backstop Disable rear wheels	11
Height adjustment	12
Folding up	12
Cleaning Maintenance	12
Technical data	13

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate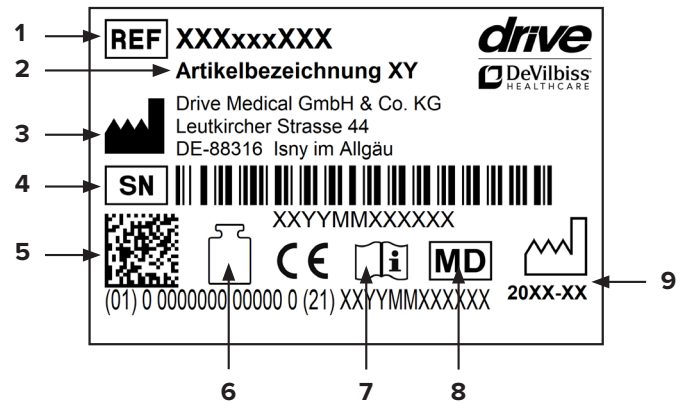

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.


Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes and handles

Parts of the product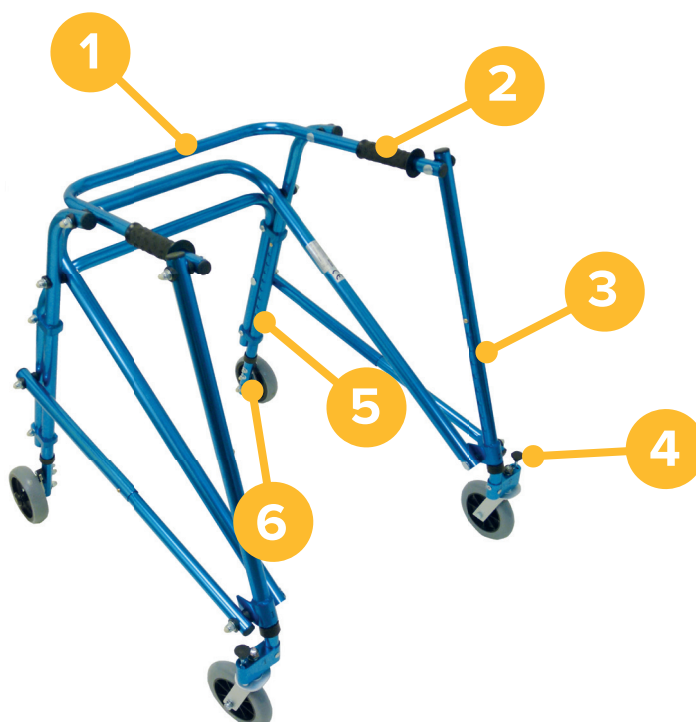

NIMBO

1. Frame
2. Handle
3. Front height adjustment
4. Front wheel lock
5. Rear height adjustment
6. Backstop

Intended purpose

Walking aids for children usually consist of a tubular frame with four rollers or wheels. They can be fitted with seats, backrests and pelvic supports and different restraint systems, which give children the freedom of movement so that they are able to move around their living area independently or with the help of an assistant.

Indication | Contraindication

INDICATION:

Walking aid for children and adolescents for use indoors to compensate for or improve limited walking ability and for correction of body posture problems.

The objective is to learn active walking or independent locomotion, to expand a limited radius of action, and to compensate for gait unsteadiness.

The children's walking aid Nimbo is pulled behind the body. As a result, the walking aid increases rear stability when standing and it offers optimal freedom of movement of the legs.

To maintain/promote/safeguard walking and standing (gait training).

CONTRAINDICATION:


Limitation of mobility in the case of impaired movement / coordination / balance – but still adequately preserved ability to walk if the use of other walking aids is not sufficient.

In general, the indication for gait training should be made under medical/orthopedic supervision. For this reason, prior to the provision of the walking aid, it should be clarified in this regard whether there are contraindications regarding the patient. Gait training aids should generally not be provided if, for example, a foot malposition and/or pain symptoms are present. If you are unsure regarding the indication and contraindication, talk to the responsible physician and/or therapist.

To avoid uncertainties regarding the indication and contraindication, we recommend consulting your attending physician and/or therapist.

Scope of delivery

1 x	Nimbo children's walking frame
1 x	Instructions for use

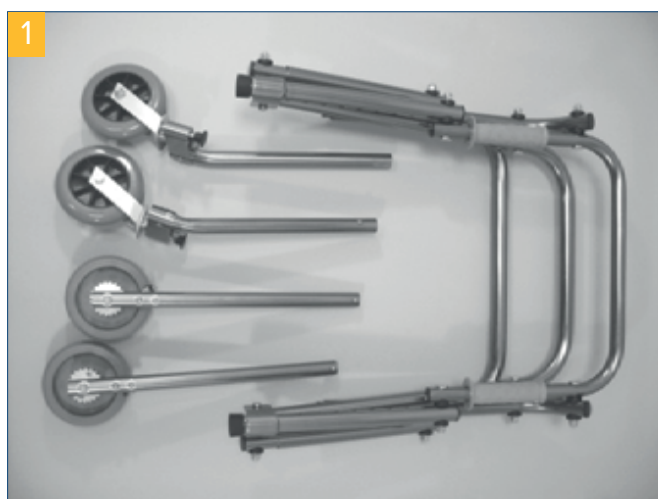
Safety instructions


- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Use the product only as a walking aid!
- Use the product only on an even and solid surface.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Use on escalators and the like is expressly prohibited!
- Do not hang any bags on the handle – risk of tipping!
- When folding and unfolding the product, clothing and body parts can be pinched.
- Protect the product from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Prior to each use, ensure that your product is intact and all adjustable parts are properly secured.
- Check the screws on the product regularly and tighten, if necessary.
- The correct use of the product requires accurate and careful training of the caregiver.
- Users who have difficulty reading must have the instructions for use read to them in order to understand how to handle the product.
- For indoor use only!
- Use only under supervision.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Preparing for use

The Nimbo has a folding aluminum tubular frame and is also height-adjustable at the wheels. To unfold it, lay the Nimbo on the ground, hold it approximately in the center of the front frame and unfold the frame. When unfolding it, be aware that the spring clips in the telescopic tubes must always properly click into place (audible click) **(photo 2)**.

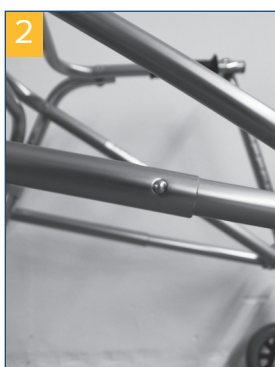
For assembly, only the front and rear wheels need to be inserted.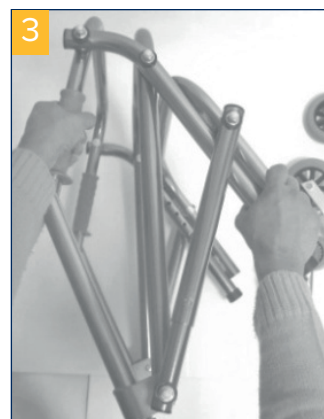

Front wheel lock

To lock the front wheels, straighten the wheels **(1)**, then pull the locking pin upwards **(2)** and rotate it 90° **(3)** until it clicks into place in the hinged plate **(4)**. To release the wheel lock, repeat the procedure.

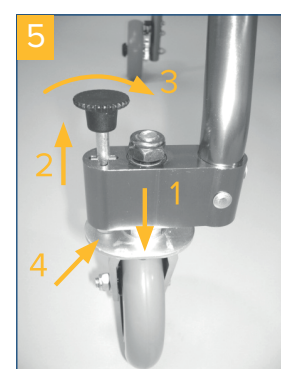Enable backstop | Disable rear wheels

Enable backstop

Connect the locking lever with the gear wheel at the wheel axle **(photo 6)**.

Disable backstop

Release the locking lever once again and push it upwards into the intended mount **(photo 7)**.


Height adjustment

The front (**photo 8**) as well as the rear wheels (**photo 9**) have 8 positions for height adjustment. To adjust the height, press the spring safety catch (arrow) and move the tubes as desired. Ensure that the spring safety catch always properly clicks into place!

Folding up

To fold up the children's walking frame, press the spring safety catch on the cross tube and then press the frame together against itself (**photo 10**).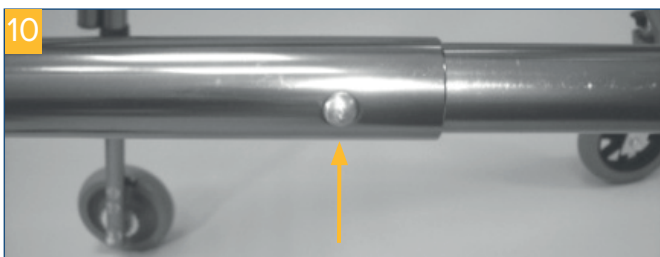

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Regularly check the metal and screw connections for wear and stability and ensure stability. Tighten screws, if necessary.
- Check in particular that the spring safety catch has clicked into place. Contact the responsible dealer if a connection is not in proper form.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Check the condition of the tires regularly. They are "fail-safe", meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.
- Necessary repairs/maintenance may be performed only by trained personnel.

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Drive Medical GmbH & Co. KG