VIPER

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Table of contents

Preface	11
Declaration of conformity	11
Service life	11
Disposal	11
Identification plate	11
Reuse	11
Warranty	11
Parts of the product	12
Intended purpose	13
Indication Contraindication	13
Scope of delivery	13
Safety instructions	13
Preparing for use	14
Hanging the footrests	14
Foot plates Calf pads	14
Length adjustment of the footrests	15
Adjusting the leg supports	15
Adjusting seat height (rear wheel mount)	15
Adjusting seat height (front wheel mount)	15
Swiveling the armrests backwards	15
Adjusting and using the parking brake	16
Anti-tipping wheels Anti-tilt aid	16
Operating the tilting aid	16
Folding up	16
Cleaning Maintenance	17
Technical data	17

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

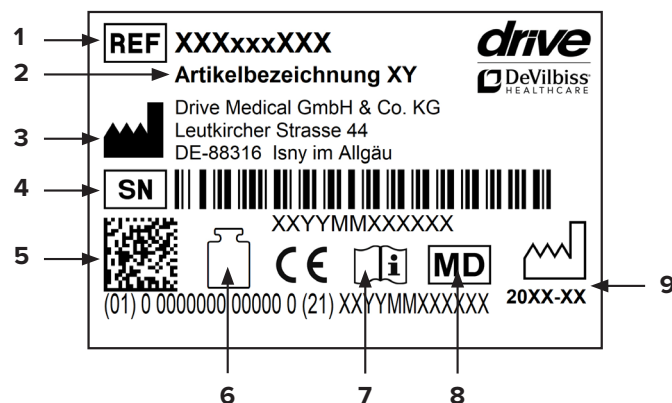1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes, armrest covers and handles

Parts of the product

Viper

- | | | |
|------------------------------|-------------------------|------------------|
| 1. Handle | 6. Anti-tipping wheel | 11. Foot plate |
| 2. Upholstered backrest | 7. Rear wheel hubcap | 12. Leg support |
| 3. Upholstered armrest cover | 8. Brakes | 13. Seat cushion |
| 4. Armrest | 9. Front steering wheel | |
| 5. Rear wheel | 10. Wheel fork | |

Intended purpose

Children's wheelchairs are exclusively for children who cannot walk or have limited mobility for individual use in everyday life, either self-propelled or with help, indoors and outdoors on firm, level ground. When supplying you must take into account the user's height, weight, physical and mental state, age, living arrangements and environment.

Indication | Contraindication

INDICATION:

Inability to walk or severely restricted walking ability as part of the basic need to move around in your own home and to leave your home to get some fresh air on a short walk or to reach the places usually in the surrounding area of your home where everyday business can be carried out.

The wheelchair exclusively serves people who are not able to walk and people who have problems with walking with their individual requirements when traveling themselves and traveling with others for everyday use, both inside and outside on even and solid ground. Additionally take body size, weight, physical and mental state, age, living arrangements, and environment into account in the provision of a wheelchair.

CONTRAINDICATION:

The use of the wheelchair is not suitable in cases of:

- Disorders of perception
- Severe loss of balance
- Loss of limbs to both arms
- Joint contractions/joint damage to both arms
- Inability to sit down

Scope of delivery

1 x	Viper children's wheelchair
1 x	Tool set
1 x	Instructions for use

Safety instructions

To ensure proper and safe use, please be sure to observe the following information:

Stability and balance

Your wheelchair is designed such that it ensures the stability you need during normal daily activities. Each of your movements in the wheelchair has an effect on the center of gravity which in the worst case can lead to tipping of the wheelchair. To increase your safety, we recommend using safety belts.

Reaching an object beyond the chair

The balance limits for reaching an object were determined through studies on a representative group of wheelchair users.

Only the arms can be extended out beyond the wheelchair seat. The body and the head should remain within the limits of the seat.

Do not lean forward

Do not lean your chest over the armrest. To reach an object in front of you, you must lean forward and bend downwards. For this reason, you must use the steering wheels as an aid (by facing them forwards) in order to attain stability and balance. Accurate orientation of the wheels is important for your safety.

Do not lean back

Position the wheelchair as close as possible to the desired object so that you can easily reach it with outstretched arms while you remain seated in the chair in a normal position. Do not lean back too far in any case since you can otherwise cause the chair to tip over.

Lateral transfer

This transfer can be performed alone, provided you have appropriate mobility and muscle tone. Position the wheelchair as close as possible to the place where you wish to sit such that the steering wheels are pointing forwards. Secure the parking brakes of the rear wheels.

Use a transfer board, if possible.

Forward transfer

If you are able to stand up, you can transfer yourself forwards to another seat. If this movement is difficult for you, we recommend obtaining assistance from an attendant or caregiver to ensure a transfer that is as easy and safe as possible.

Secure both brakes and ensure that both steering wheels are pointing forwards. In doing so, be aware that you should not under any circumstances support yourself on the brakes during the transfer. Then grasp the top of the armrests, support yourself on your arms in order to easily get up and the attendant can assist you in completing the transfer. Position yourself as near as possible to the place where you wish to sit.

During the transfer, position yourself as far back in the seat as possible to prevent the seat cushion from being damaged or the wheelchair from tipping forward.

Secure the two brakes; the brake handles should never be used for support during transfers. Never support yourself on the foot plates when getting up from the wheelchair.

If you are able to stand more or less upright and if you have sufficient upper body strength and mobility, you can transfer yourself forwards to another seat. Fold up the foot plate and push the foot support/leg support to the side to clear the way, bend your body forwards while you support yourself on the two armrests and stand up. Then push your body in the direction of the place where you wish to sit while distributing your weight on your arms and hands.

Overcoming obstacles with an attendant

Getting up onto the sidewalk

The attendant positions the wheelchair facing forward in front of the sidewalk. He/she then tilts the wheelchair backwards towards him-/herself until the steering wheels reach up onto the sidewalk; he/she pushes the wheelchair forward until the rear wheels are at the edge of the curb and pushes the wheelchair further until the rear wheels roll over the edge of the curb.

Getting down off the sidewalk

The attendant positions the wheelchair facing forward with the steering wheels at the edge of the curb and tilts it backwards towards him-/herself. Then he/she carefully pushes the wheelchair forwards until the rear wheels touch the street, then he/she gradually lowers the steering wheels to the ground.

Getting over stairs

Since it is difficult to get over stairs, we recommend getting assistance from two attendants, one in front of the wheelchair and one behind the wheelchair at the push handles. The attendant who is positioned behind the wheelchair tilts it backwards towards him-/herself while holding the handles firmly. The attendant who is positioned in front of the wheelchair holds the front lateral frame firmly and lifts the chair over the stairs while the other person places one foot on the next stair and repeats the step.

Do not lower the steering wheels of the wheelchair until the attendant positioned in front of the wheelchair has crossed the last stair. To go down stairs, the same procedure as described above should be followed, in reverse order.

Lifting the wheelchair with hoists

Do not attempt to lift the wheelchair by grasping any removable parts (for example, the armrests, leg supports or foot supports). Avoid using an escalator.

Traveling on inclines by independent wheelchair users

We recommend avoiding ramps with an incline of more than 9°. In the case of larger inclines, there is a risk of tipping over if the wheels spin or when approaching an incline at an oblique angle.

Going up ramps

Lean your upper body forward and operate the hand rims with rapid pushes to reach an appropriate speed. If you want to stop to rest along the way, the two brakes must be operated simultaneously.

Going down ramps

Lean your upper body backwards and use the hand rims which slide through your hands to control the speed and direction. Avoid sudden direction changes, never attempt to ascend or descend a ramp diagonally, and do not attempt to turn the wheelchair when ascending. The

wheelchair is not suitable as a seat for driving a vehicle. Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.

Protect the product from direct sunlight

Product parts can become hot and cause skin injuries if touched.

Note

The wheelchair is exclusively used to transport the person for whom it has been adapted by a qualified dealer. For children or persons with impaired decision-making ability, parents or authorized caregivers must ensure that the information in the instructions for use is followed. The safest use of the aid is possible only on even, solid and dry ground with tilt protection enabled and with the aid of an assistant.

Preparing for use

The Viper children's wheelchair is delivered completely assembled. Some simple maneuvers will make it ready for use.

Risk of pinching!

During folding or unfolding, clothing and body parts can be pinched.


Hanging the footrests

Hang both footrests provided as shown and swivel them inwards until they audibly click into place (**photo 1**).


Foot plates | Calf pads

After you have inserted the footrests, rotate the foot plates downward. You have the option of adjusting the calf pads in three heights as desired. You can change the position of the calf pads without tools upwards or downwards (**photo 2**).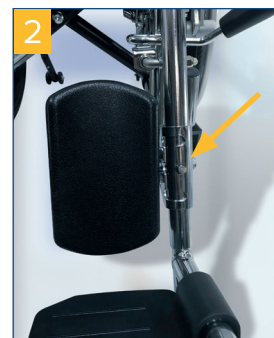

Length adjustment of the footrests

Loosen the star screw on the underside of the footrest (**photo 3**) and adjust the length as desired by pushing up or down. Retighten the screw!

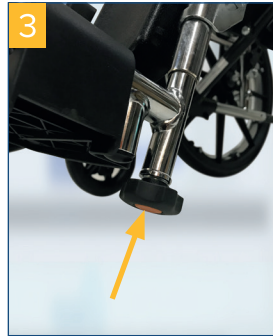

Adjusting seat height (rear wheel mount)

The height of the seat at the rear can be adjusted in 2 heights. Remove the screw of the wheel axle and remove the axle. Mount the wheel at the desired height and screw the screw back in. Ensure that the wheel axle is properly tightened (**photo 8**)!

Adjusting the leg supports

To take the leg supports off, first raise the foot plates. Press the locking lever and rotate the leg support to the side (**photo 4+5**). Pull the leg supports up and out. To mount them, perform the operation in the reverse order. Ensure that the leg supports are correctly locked.

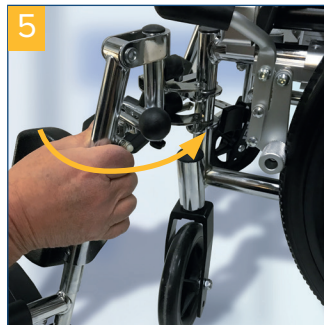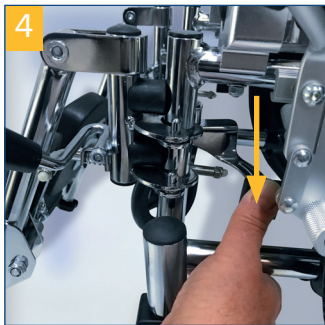Raising the leg supports

To raise the leg supports, simply pull the leg support upwards into the desired position. The leg support locks by itself in the adjusted position (**photo 6**).

Lowering the leg supports

To lower the leg support again, pull the lever and swing the leg supports down (**photo 7**).

Adjusting seat height (front wheel mount)



The height of the seat can be adjusted in increments and is correspondingly adjusted in the desired inclination of the seat surface. To adjust, unscrew the wheel axle screw and pull it out. Now move the wheel in the desired position and reinsert the wheel axle screw and tighten it (**photo 9**).

Ensure that the wheel axle is properly tightened!

Swiveling the armrests backwards

To swivel the armrest backwards, engage the release lever at the front of the armrest and hold it down (**photo 10**) while simultaneously pulling the armrest up until the armrest is free. Now it can be swiveled backwards (**photo 11**).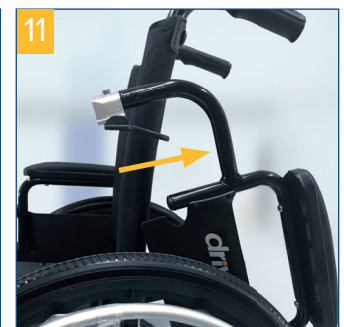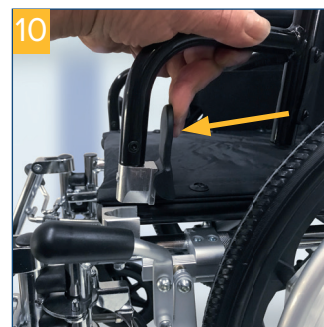

Adjusting and using the parking brake

Upon delivery, the parking brake is correctly adjusted according to the configuration status. If you have made changes to the wheel position, the brake adjustment may need to be changed.

To do this, remove both screws on the upper side of the respective brake and push the complete unit onto the guide such that there is a distance of 10 mm between the brake bolt and the tire. Now retighten both screws. The parking brake is now optimally adjusted (**photo 12**).

Ensure that all screws are reinserted and tightened!

To use the parking brake, press the brake lever firmly to the front until it noticeably clicks into place. The parking brake is now engaged.

To release, pull the brake lever towards you and the parking brake is unlocked (**photo 12**).

Always ensure that the parking brake is locked by attempting to move the wheelchair.

Anti-tipping wheels | Anti-tilt aid

The anti-tipping wheels prevent the wheelchair from tipping over backwards. They also serve to facilitate tilting by an attendant. The height of the wheels can be adjusted. Press both silver-colored buttons (front and rear) and slide the tubing along the bracket up or down. By pulling the anti-tilt aid up, you can pivot it upwards and away (**photo 13+14**).

Operating the tilting aid

To maneuver around curbstones and over thresholds, step firmly (with the left or right foot) on the tilt aid while simultaneously pushing the push handles down. This ensures that lifting the front of the wheelchair is easier. Ensure that you are standing securely (**photo 15**).

Folding up

The wheelchair can be folded up for easier transport or for stowing in motor vehicles. To do this, first remove the optional seat cushion as well as the footrests. Then hold the side of the seat surface cushion (**photo 16**) and pull the seat surface upwards (**photo 17**).

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.)
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Check the condition of the tires regularly. They are "fail-safe," meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

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Drive Medical GmbH & Co. KG