INSTRUCTIONS FOR USE

Your doctor or other healthcare professional has prescribed the OXYMIZER oxygen-conserving device for you. The OXYMIZER helps conserve oxygen by means of a reservoir system, which allows for the accumulation of oxygen during the exhalation and pause phases of a person's breathing pattern. Used properly, this device will provide you with the oxygen that you need, along with some additional benefits.

It is important that you follow your physician's instructions carefully concerning the use of the OXYMIZER:

- The OXYMIZER is designed to replace your standard nasal cannula and is designed for use with continuous flow oxygen systems.
- Use of the OXYMIZER may allow you to reduce the flow of oxygen delivered by your oxygen system (i.e., cylinder, oxygen concentrator and standard nasal cannula) without reducing the amount of oxygen delivered into your body. The reduction in continuous oxygen flow can be as much as 75% less than when using a standard nasal cannula; this can vary based on the liter flow and the user's breathing pattern.
- If you use the OXYMIZER with a portable oxygen system (i.e. cylinder), the reduction in flow will generally allow the system to last longer. This may allow you to be away from your main oxygen source for a longer period of time and/or may enable the use of a smaller, lighter weight portable system.

IT IS IMPORTANT THAT YOU FOLLOW YOUR PHYSICIAN'S INSTRUCTIONS CAREFULLY CONCERNING:

- The details of your oxygen prescription, including when and how long to use per day.
- The oxygen liter flow rate to use.

NOTE: The flow rate indicated for use with the OXYMIZER device may be one quarter to one half of the oxygen liter flow prescribed for use with a standard nasal cannula.

- When to replace your OXYMIZER device.

HOW TO PUT ON THE OXYMIZER PENDANT

1. Uncoil the tubing completely. Hold the device so that the front of the Pendant is facing away from you.

NOTE: The front of the Pendant is designated with slots.

2. Position the nasal prongs so that they fit comfortably into your nostrils. The prongs should extend well into and point toward the back of your nose.

3. Position the nasal prongs and extend the tubing across your cheekbones.

4. Place the tubing around your ears and adjust each side so that you have a snug, comfortable fit. Do not allow the tubing to go completely around your neck. This may create a choking hazard.

5. Attach the tubing clip to join both sides of the cannula tubing. Slide the clip upward beneath your chin to hold the tubing in place (you may choose not to use the clip simply by removing it). Do not over-tighten to avoid injury.

6. The Pendant portion should rest against your chest below your throat. You may want to tuck it under your shirt or blouse.




7. Connect the female connector to your oxygen source.

8. Turn on the oxygen and set the flow rate as prescribed by your doctor or health care professional.
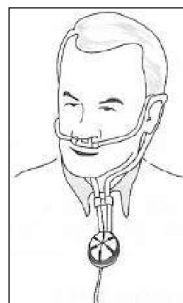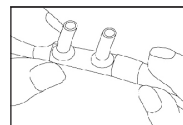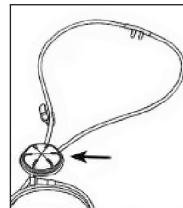

NOTE: The flow rate prescribed for use with this device may be one quarter to one half that prescribed with a non-oxygen-conserving cannula.

WARNING: This device is intended for single patient use only and can be used multiple times by the same patient. Sharing this device with other users may increase your risk of respiratory infections or other communicable diseases.

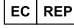CAUTION: The use of nasal oxygen delivery devices, such as nasal cannulas may increase the risk of skin irritation and/or breakdown on the ears, face and opening of the nares.¹

SYMBOL DEFINITIONS

	No Smoking		Keep Dry		European Representative
	Keep away from open flame		Prescription Only		CE Marking of Conformity
	Catalog Number		Lot Number		Manufacturer

REPLACEMENT

The OXYMIZER Pendant device should be replaced after approximately three weeks. More frequent replacement may be needed for cleanliness and sanitation, depending on conditions of use.

TODAY'S DATE:	REPLACE ON:

Contact your healthcare provider or the manufacturer to report unexpected operation or events.

SUPPLEMENTAL HUMIDIFICATION

Supplemental humidification is frequently recommended for use with a nasal cannula. This is typically accomplished by placing an appropriate oxygen humidifier between the oxygen source and the cannula. Since the OXYMIZER cannula uses significantly less flow of oxygen than standard nasal cannulas, the need for supplemental humidification may be reduced or eliminated. Follow your healthcare professional's recommendations regarding the use of humidifiers.

CLEANING

The disposable OXYMIZER oxygen-conserving devices are clean and ready to use, and are supplied in individual containers. The outer surface may be cleaned as needed with soap and water, but care must be taken not to allow the membrane within the reservoir to become wet.

INTENDED USE

The device is intended to be used as part of a portable or stationary oxygen system for patients that require supplemental oxygen in their home and for ambulatory use.

CONTRAINDICATIONS

- Not for use by infants and small children
- Not for use by patients with nasal blockage
- Not for use by patients with facial injuries that would preclude the use of a cannula
- Not for use by patients who will not leave the nasal cannula in place
- Not intended for multiple patient use
- Not for use with pulse dose oxygen delivery systems

DISPOSAL

Disposal for your device should be done in accordance with local regulations.

SPECIFICATIONS

Max. Inlet Pressure	345 kPa (3.45 bar, 50 psi)
Max. Flow (LPM)	15 LPM
Operational Temp.	5°C to 40°C (41°F to 104°F)
Operational Humidity	15% to 93% non-condensing
Storage Temp.	-25°C to 70°C (-13°F to 158°F)
Storage Humidity	<= 93% non-condensing

Expected Service Life is 21 days.

WARNING: To avoid strangulation keep tubing away from children, pets, and closely supervise impaired persons. Do NOT route tubing around neck.

OXYMIZER[®] PENDANT
DISPOSABLE OXYGEN-CONSERVING DEVICES

 P-224

FOR USE IN OXYGEN THERAPY

DISPOSABLE-FOR SINGLE PATIENT USE ONLY.
INSTRUCTIONS FOR USE ENCLOSED

CONTENTS: One (1) OXYMIZER Pendant disposable oxygen-conserving device with curved tubular nasal prongs, six (6) feet of connecting tube, a female connector and user's manual.

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Inovo, Inc • 401 Leonard Blvd. North
MDSS GmbH • Schiffgraben 41