

Suction Container Cleaning Instructions



1

Turn the suction unit "off" and allow vacuum to drop.



2

Disconnect the suction unit from its power source.



3

Disconnect the tubing from the canister and remove from holder.



4

Carefully remove lid and empty contents.
Container should be emptied and cleaned after each use.



5

Remove filter cartridge and 4-1/2" (11.43 cm) tubing and set aside.

Do not remove floatball from lid. If removed, floatball may pose a choking hazard.



Filter Cartridge MUST NOT get wet.

The filter material cannot be removed from the elbow.

Change filter cartridge if overflow occurs or every two months, whichever comes first.



6

Wash container and lid in warm water/dishwashing solution. Rinse with clean, warm water.



7

Soak in 1 part vinegar to 3 parts warm water solution for 60 minutes. Rinse with clean, warm water and air dry.